
magno[®] **DIGITAL**

by Eschenbach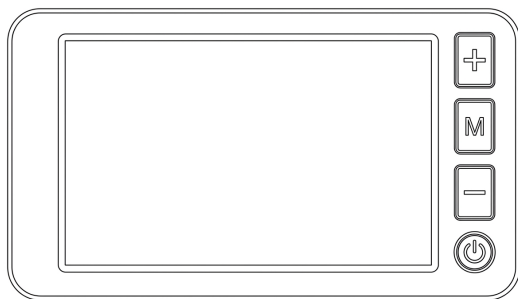

Bedienungsanleitung	DE
Operating instructions	EN
Mode d'emploi	FR
Istruzioni d'uso	IT
Manual de instrucciones	ES
Gebruiksaanwijzing	NL
Betjeningsvejledning	DK
Bruksanvisning	SE
Bruksanvisning	NO
Käyttöopas	FI
Instrukcja obsługi	PL
Návod k obsluze	CZ
取扱説明書	JP

Abb. 1

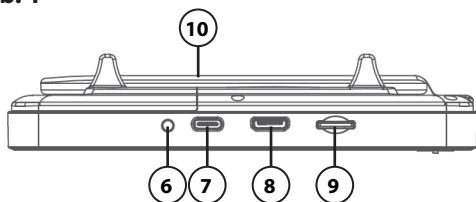
Abb. 2


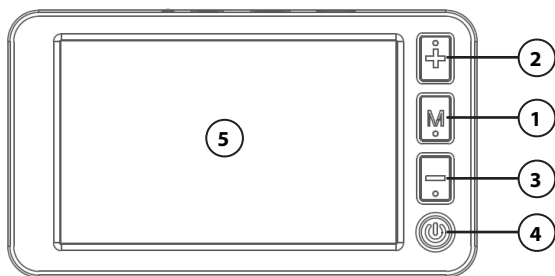
Abb. 3

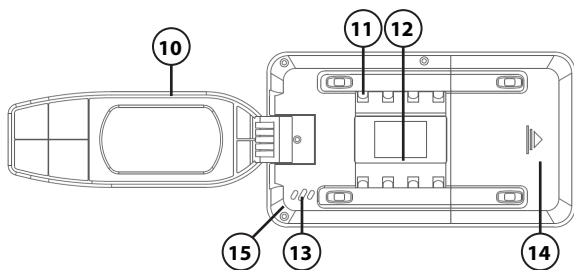Funktionselemente

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
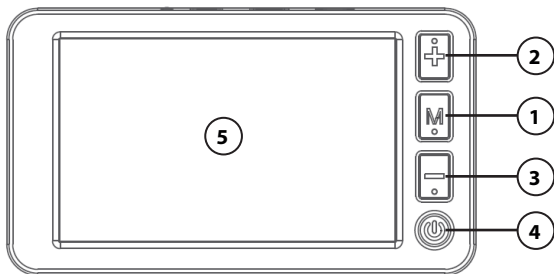
Fig. 3

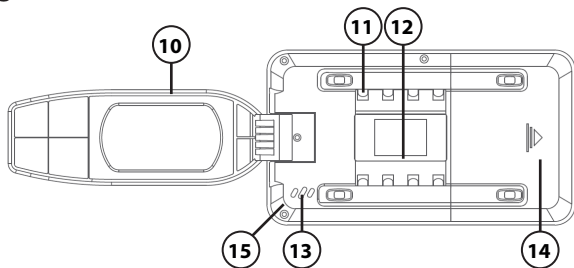Operating elements

- | | |
|------------------------------------|---|
| 1. Contrast Button and Menu Button | 9. TF Card Slot |
| 2. Increase zoom factor | 10. Handle |
| 3. Decrease zoom factor | 11. LED illumination for reading material |
| 4. On/Off button | 12. Camera |
| 5. Display | 13. Speaker |
| 6. Charging indicator LED | 14. Battery cover |
| 7. Charging socket (USB-C) | 15. Eyelet for neck strap |
| 8. HDMI output (Mini-HDMI, Type C) | |

Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images.

Target group

magno DIGITAL is designed for people with low vision.

Indications / Contra indications

Visual impairment in which adequate reading ability can no longer be achieved with conventional optical aids (e.g., magnifying glasses, eyeglasses) and an electronic magnifying aid is necessary for daily living. No known contra indications.

Before first use

Before using the device for the first time, please read the entire manual. This is the only way to get to know all the functions on the device.

Safety instructions

- ▶ Risk of glare and injury: Do not look into the light sources on the underside of the housing.
- ▶ Do not allow children to handle electronic devices without supervision.
- ▶ Only charge the reader with the mains adapter supplied.
- ▶ Choking hazard and danger of swallowing: Keep packaging, foils and packaging parts away from children.
- ▶ Check the device and the mains adapter with power cable for visible damage. In the event of damage, disconnect the connected mains adapter from the power supply immediately. Ask the manufacturer or its customer care to check the product before you use it again.
- ▶ Lay the mains cord so that nobody can trip over it.
- ▶ Never open the device housing. There are no serviceable parts whatsoever inside. Opening it will invalidate the warranty.
- ▶ The installed lithium-ion batteries may only be removed or replaced by the manufacturer or its customer care.
- ▶ **Caution** Risk of explosion! Never replace the battery with a wrong type. Never throw a battery into fire. Never crush or cut a rechargeable battery. Never leave a battery in an environment with extremely high temperatures, as this may lead to an explosion or the leakage of flammable liquids or gases.
- ▶ Never expose a battery to extremely low air pressure, as this can lead to an explosion or the release of flammable liquids or gases.
- ▶ Never expose the device to moisture.
- ▶ Risk of trapping: When folding the handle, be careful not to pinch your fingers.
- ▶ Protect the device from impact or shock and excessive heat.
- ▶ Never place the device on radiators or in direct sunlight.
- ▶ Do not store the device in places with excessive heat, such as in a parked car.

- ▶ Only intended for use in a domestic environment.
- ▶ Do not use the device while it is charging.
- ▶ In medical environments, this device must not be operated in conjunction with other life-support equipment.
- ▶ Please also alert other people - particularly children - to this danger.
- ▶ The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. When using the device outside the EU member states, the respective country-specific directives must be observed.

Scope of delivery

- ▶ magno DIGITAL
- ▶ Charger for US
- ▶ Operating instructions

Basic operation

Before you can use magno DIGITAL for the first time, the internal battery must be fully charged.

Connect the included USB cable to the USB port of the power adapter. Now connect the USB cable to the device's ⑦ charging port. Then plug the power adapter into the power outlet.

While the battery is charging, the charging indicator LED glows ⑥ red. When the battery is fully charged, the LED turns off.

During the charging process, ensure that the charger and the magno DIGITAL have adequate heat dissipation and ventilation to prevent hazards caused by overheating.

Switching on/off

Press the -button ④ for one second to turn on the device. After a few seconds the camera image will appear on the display ⑤.

To turn off, press and hold the **⏻**-button ④ for one second. The current settings are saved.

Application types

You can use the device with or without ⑩ handle. The handle ⑩ is located on the back of the device and can be unfolded (see Fig. 3 on page 2).

View functions

Adapting the zoom

To increase the magnification while displaying the live image, press and hold the **+**-button ②.

To decrease the zoom factor, press and hold the **-**-button ③.

The currently set magnification factor is displayed in the top right corner.

False-color representation

To display the camera image in false colors for contrast enhancement, press the **M**-button ①. Press this button again—repeatedly, if necessary—to cycle through the various false-color modes and return to the true-color display.

The false colors are preset to black/white and white/black as delivered.

Freeze Live View

Briefly press the **⏻**-button ④ to freeze the current camera image. In this image as well, you can change the magnification factor and the contrast color, as previously described.

Press the **⏻**-button ④ again to return to the live image display.

Saving the live image as a file

Hold down the **—**-button ③ and then briefly press the **⏻**-button ④. The photo has been saved, and a number appears in the top left corner. Hold down the **—**-button ③ and then briefly press the **M**-button ① to open the photo album. To return to the live view, press and hold the **—**-button ③ and then briefly press the **M**-button ①.

In the photo album, the keys are assigned as follows:

Buttons	Short button press	Long button press
M -button ①	Change contrast colors	Open photo menu
+ -button ②	Zoom in photo	Zoom in photo
— -button ③	Zoom out photo	Zoom out photo
⏻ -button ④	Next photo	Turn the device on and off

HDMI output

You can connect the device from its HDMI output ⑧ to the HDMI input of another device, such as a TV.

Timer shut-off

If no button is pressed and the screen remains inactive for one minute, the device automatically enters sleep mode, and the screen displays the battery level as well as the current date and time. In sleep mode, you can wake up the device by pressing any key.

Info: If no operation occurs after 2 minutes, the device automatically switches off to save power.

Settings

Keep the **M**-button ① to open the menu. A yellow-and-black screen appears. After 10 seconds, the device switches back to the live view if

no button has been pressed.

You can adjust the setting using the **+**-button ② and **-**-button ③. Other setting options can be selected using the **M**-button ①.

Adjust contrast colors

Adjust display brightness

In this example, you can individually adjust the display brightness from 1 to 5 using the **+**-button ② and **-**-button ③. A brief press of the **M**-button ① takes you to the next setting option:

Adjust LED Lighting

You can individually adjust the LED lighting from 0 to 5 using the **+**-button ② and **-**-button ③. At setting 0, the LED lighting is completely switched off.

Adjust volume


You can individually adjust the LED lighting from 0 to 5 using the **+**-button ② and **-**-button ③. At setting 0, the sound is switched off.

Adjust automatic shut-off time

You can use the **+**-button ② and **-**-button ③) to individually set whether the device switches off automatically after 3 minutes, 10 minutes, or 30 minutes. If set to 0m, the device will not turn off automatically.

Setting the date and time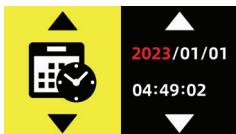

You can set the date and time using the **+**-button ② and **-**-button ③. With a brief press of the **⏻**-button ④, you can move from left to right and adjust the next digit.

Adjust contrast colors

Different contrast colors can be cycled through using the **+**-button ② and **-**-button ③. To enable or disable the individual contrast colors, briefly press the **⏻**-button ④.


An activated contrast color is marked with a green check mark. If no green check mark is visible, the contrast color is disabled. Only the activated contrast colors can be selected in the Live View using the **M**-button ①.

Battery level


The battery level is displayed in percent. When the device is switched on, a battery icon displaying the battery level appears in the centre of the display and in the bottom right corner.

Photo album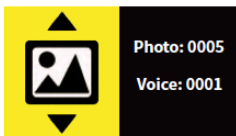


The number of photos in the photo album is displayed here.

Storage space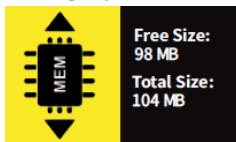

The available storage space is displayed.

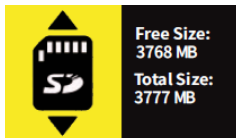When a TF card is inserted into the TF card slot, ⑨ this view appears.

Firmware

The current firmware version can be viewed here.

Photo album menu

In the photo album, you can access the submenu by long-pressing the **M**-button ①.

The following appears:

In this view, you can delete the selected photo. The number **1** next to the trash can indicates this. Press the **+**-button ② or **-**-button ③ to select the photo to delete.

The following appears:

The red check mark indicates that the photo has been selected for deletion. You can confirm the deletion of the photo using the **U**-button ④.

Delete All Photos

If you would like to delete all photos in the photo album, select this icon:

Press the **+**-button ② or **-**-button ③ to delete all photos. A red check mark indicates that all photos can be deleted. You can confirm the deletion of all photos using the **U**-button ④.

Info: Once the photos have been deleted, it is no longer possible to recover them.

Formatting the storage

If you wish to format the storage, select this icon:

Press the **+**-button ② or **-**-button ③ to format storage. A red check mark indicates that all photos can be deleted. You can confirm formatting by pressing the **U**-button ④.

Info: As soon as the hard drive has been formatted, all files are deleted and can no longer be recovered.

Technical specifications	
Display size:	12.7 cm (5")
Zoom range:	3x – 48x
Display modes:	True color and 25 selectable contrast colors
Operating time:	approx. 4 hours
Battery:	Lithium-ion battery, 2500 mAh, 3.7 V
USB connection:	Type-C
Temperature range:	10 °C – 40 °C (50–104 °F)
Dimensions:	151 x 86 x 31 mm (W x D x H) 5.94 x 3.39 x 1.22 in (W x D x H)
Weight:	220 g

USB® is a registered trademark of USB Implementers Forum, Inc. HDMI and the HDMI Logo are trademarks or registered trademarks of HDMI Licensing LLC in the United States and other countries.

Care instructions

- ▶ When cleaning the device do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents!
- ▶ Do not clean the device in an ultrasonic bath or under flowing water!
- ▶ Only use a soft, dry cloth to clean the device. If the optical elements (camera, LEDs) are soiled—for example, with fingerprints or similar marks—clean them carefully with a microfibre cloth.

Disposal

- This product is subject to the European WEEE directive on old electrical and electronic equipment in its currently valid version. Do not dispose of your old device with household waste, but via an authorised waste disposal company or your municipal waste disposal facility.

Observe the regulations applicable in your country. If in doubt, contact your waste disposal organisation. Dispose of all packaging materials in an environmentally friendly manner.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty only by proof of purchase.

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Trade Name: magno DIGITAL

Model No.: 916505

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ▶ Reorient or relocate the receiving antenna.
- ▶ Increase the separation between the equipment and receiver.
- ▶ Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ▶ Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

UK Importer:

Associated Optical Products
13 Gore Road
SL18AA Burnham, Bucks
Internet: www.associatedoptical.com

Fig. 1

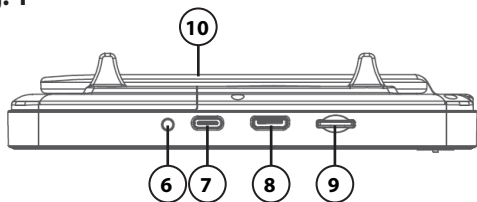
Fig. 2

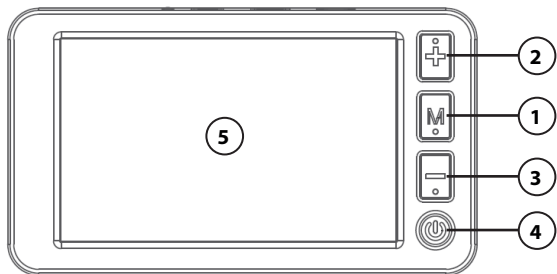
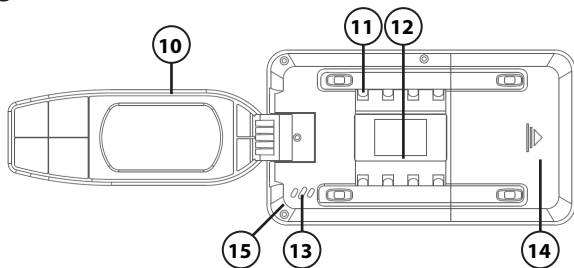
Fig. 3

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Eschenbach Optik GmbH