# vario **DIGITAL** 22 FHD

16592

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Operating instructions **EN**

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Manual de instrucciones **ES**

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Bruksanvisning **SE**

Bruksanvisning **NO**

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Dear Customer,

congratulations on the purchase of your Eschenbach vario DIGITAL 22 FHD.

As the market leader for magnifying glasses and reading aids, we at Eschenbach can look back over many years of experience and this is reflected in the quality, usability and design of our products.

## **Firmware version**

This manual is based on firmware version "VD22\_fw\_1.0" or higher.

## **Intended use**

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images. To guarantee correct functioning, the screen reader must be set up as described in the assembly instructions.

The product is operated in a domestic environment (also includes nursing homes, workplaces, schools and other educational facilities).

## **Before first use**

Please read these operating instructions thoroughly before using the device for the first time. This is the only way to get to know all the features available for the device.

## **Package contents**

- ▶ vario DIGITAL 22 FHD
- ▶ Power adapter
- ▶ Power plug adapter for EU, GB, US, AU
- ▶ USB cable (Type C)
- ▶ Microfiber cloth
- ▶ Operating instructions

## Safety information

EN

- ▶ **Risk of damage to the eye and risk of injury: Never look into the light sources on the underside of the device.**
- ▶ **Never allow children to handle electric devices without supervision.**
- ▶ **Risk of suffocation and risk of accidental swallowing: Keep packaging, films and packaging parts away from children.**
- ▶ **Risk of pinching: Ensure that your fingers do not get pinched during unfolding and folding of the device, and also during unfolding and folding of the camera on it.**
- ▶ **The entire device must stand on a stable surface.**
- ▶ **Take care not to leave any objects on the base when you want to fold the device together. This can result in damage to the display.**
- ▶ **Protect the device against impact or shock. Handle the device with care.**
- ▶ **Never subject the device to humidity or moisture.**
- ▶ **Protect the device from excessive heat and open flames. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in places exposed to excessive heat, such as in a parked car.**
- ▶ **Use only the supplied power adapter and country-specific power plug adapter.**
- ▶ **Check the device, the power adapter and its cable for visible damage. If there is any damage, disconnect the power adapter from the power supply immediately. Have the product checked by the manufacturer or its customer service department before using it again.**
- ▶ **Route the power cord so that it does not present a tripping hazard.**
- ▶ **Class 1 laser in acc. w. EN 60825-1:2014.**
- ▶ **Never open the device housing. Opening the housing voids the warranty.**

- ▶ **The device must be turned off and the power plug must be removed before it is folded together.**
- ▶ **Carry the device only when it is folded together and by using the carrying handle.**
- ▶ **For domestic use only.**
- ▶ **In a medical environment this device may not be operated together with other devices that are used for life support.**
- ▶ **Ensure that other people, especially children and persons who cannot assess this hazard, are aware of this hazard.**
- ▶ **The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the device outside the EU member states, the respective country-specific directives must be observed.**

## Functional elements

(For illustrations see pages 2 to 4)

EN

1. Display
2. Rotary knob to open contrast functions
3. Photo button for saving images and confirming menu options
4. Button for turning the reading line on and off
5. On/Off button
6. Button for opening menu operation
7. Button for overview display
8. Rotary knob to increase or decrease zoom factor
9. Release button on the base arm
10. SD card (insert card as shown)
11. SD card slot
12. USB port
13. RESET button for resetting the device
14. Monitor arm
15. Button for moving to the right in menu
16. Button for moving to the left in menu
17. Button for moving upward in menu
18. Button for moving downward in menu
19. Foldable camera cover (flipped open in illustration)
20. Camera
21. Base plate
22. Release button on the monitor arm
23. Base arm for unfolding the device
24. Charging control LED
25. Connection for power adapter
26. Carrying handle

## Basic operation

### Structure of the device and power supply

To avoid damage to the product, follow the assembly instructions, which can be found in the product packaging. Always set up the device on a suitable surface only, e.g. a stable table.

The camera ⑳ is equipped with a cover ⑲. Open the camera cover and move it to the designated position (see illustration on fold-out pages A1 and A2). If the  icon is displayed, the camera cover has not been opened.

You can adjust the height of the screen and its tilt angle on the monitor itself.

Connect the power adapter to a suitable device plug. Connect its cable to the connection for the power adapter ㉕. Then plug the power adapter into the power socket.

The charging control LED ㉔ lights up while the device is being charged.

To fold the device, follow the assembly instructions, which can be found in the product packaging. Keep the assembly instructions for later use.

### Turning on/off

Press the On/Off button ⑤. The camera icon will appear on the screen ① after a few seconds. Icons briefly show the current status, for example magnification factor and brightness setting.

To turn the device off, press and hold the On/Off button ⑤ for one second. Current settings are saved.

### Reading with the device

Turn the device on and place the reading material on the base plate ㉑ under the camera ㉒.

In exceptional cases it can make sense to relocate the field of vision to the upper area of the base plate. To do this, swivel the camera out past the locking position. It is also possible to move the camera forward, for example for carrying out handicrafts.

## Adapting the zoom

To increase the magnification turn the rotary knob ⑧ in clockwise direction when the live image is displayed. To decrease the magnification, turn the rotary knob ⑧ counterclockwise.

The magnification factor you have set will be displayed briefly.

## Overview function

If you would like to get an overview of the reading material, press the  button ⑦. The magnification changes to 1.6 ×. If you press the  button again, the previously set magnification will be selected again.

## Turning the reading line/row on and off

Press the ④ button to turn the reading line on. Press it again to hide the line.

## Special functions of the buttons

- ▶ Press and hold  button ③ to open the photo album.
- ▶ Press and hold  button ⑦ to turn the autofocus function off (see also Chapter “Deactivating/activating autofocus” on page 27).
- ▶ Press and hold the reading line button ④ to change the reading material brightness (see Chapter “Adjusting reading material brightness” on Page31).

## False color display

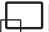
To display the camera image in false colors for increased contrast, press the rotary knob ② until the required false color rendering is displayed. By default, the false colors are preset as black/white and white/black. To adapt the false color display, open the „Adapt false colors” menu (see Page 31).

## Saving the live image as a file

Press the  button ③ briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the screen. You can also change the magnification factor for this image as already described. You can use the arrow keys ⑮ + ⑯ to

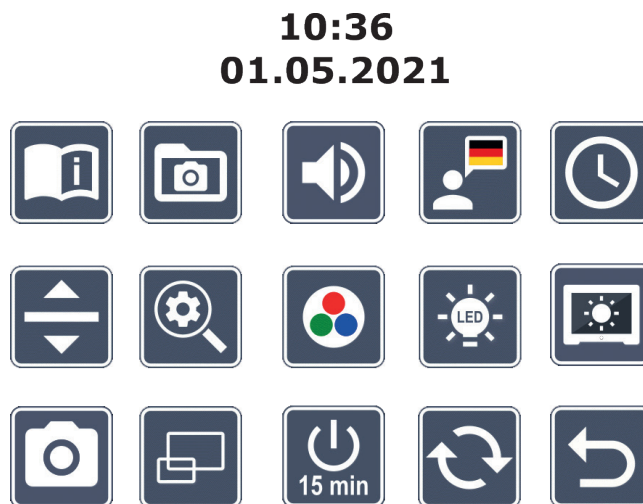
move the magnified image section vertically and arrow keys 17 + 18 to move it horizontally. Press the Photo button again to return to the live image display.

### Deactivating/activating autofocus

Press and hold the  button 7 to deactivate the autofocus function. This can be helpful, for example while writing under the device, so that the focus remains on the reading material. Press the  button again to reactivate autofocus. Autofocus is automatically reactivated when the device is turned off.

### Opening the main menu

Press the MENU button 6 to open the menu:

You can navigate to the individual icons using the rotary knobs 2 + 8 and the arrow keys 15, 16, 17, 18. The icon for the selected function is displayed slightly larger and in color. To activate the function you have selected, press the  button 3.

Individual menu commands are also played back acoustically. Playback can also be deactivated by changing the volume. To do this, press the  icon several times until the  icon is displayed.

If you press the  button 7, the currently selected menu item is audibly played back again.

You can exit the setting menu at any time by simply pressing the MENU button ⑥.

EN

### Opening the quick guide

Select this icon and press the  button ③ to access the quick guide stored in the device. The following toolbar appears:

Select the national flag that represents the language you desire. Press the  button ③ to change the language. You can use  /  to scroll the page preview backward and forward. The operation is analogous to the photo library.

### Photo library

Select this icon and press the  button ③ to manage image files on the SD card. After accessing the photo library, the last image taken is displayed centrally. On the left and right you can see small sections of the previous and next image. The following toolbar is visible:

Using the middle icon you can hide the other icons and display the middle photo as a full screen.

The number of the currently selected image file and the total number are shown at the bottom left of the display as (X / XX).

With the toolbar displayed, you can navigate to the individual icons using the rotary knobs ② + ⑧ and the arrow keys ⑮, ⑯, ⑰, ⑱. The icon for the selected function is always shown slightly larger. To activate the following functions, press the  button ③:

-  - delete the selected image from the SD card
-  - select the previous image
-  - If this icon is highlighted and confirmed with the  button ③, the navigation bar is hidden.
-  - select the next image
-  - back one level

When the toolbar is hidden, the rotary knobs ② + ⑧ and the arrow keys ⑮, ⑯, ⑰, ⑱ function the same as during live image display. To show the toolbar again, press the  button ③.

## Deleting image files

To delete the selected image file from the SD card, first select the delete icon  and then confirm with the  button ③. The confirmation icon  is now displayed on the bottom right of the screen, and the cancel icon  is displayed on the left. You can use the rotary knobs ② + ⑧ to select  or . The icon you have selected will be displayed slightly larger. Press the  button to execute the selected function.

## Adjusting volume

The current volume setting is displayed. Select this icon and then press the  button ③ to select the following settings:

- very low volume

- low volume

- medium volume

- high volume

- mute

## Selecting the language for voice output

EN

Select this icon and then press the  button  to select the language for voice output. The icon for the current voice output setting is displayed. Alternatively you can select  or  for confirmation using a signal tone.

The  icon is displayed while the language file is being loaded.

## Setting the date and time

The time will only be displayed in the menu once it has been set. Select this icon and then press the  button  to set the time. Select the  icon so that you can increase the number of the parameter displayed in yellow using the  button . Select the  icon and then use the  button to decrease the number of the parameter displayed in yellow. Select the  icon and then press the  button to select the next parameter for setting.

Select „24H“ and then press the  button to switch between 24-hour display and the 12-hour display usually used in English-speaking areas.

The date format normally used in the EU is set as standard. Select EU and use the  /  icons to change the date format if required.

## Setting the reading guideline/line

Select this icon and then press the  button  to adjust the details of the reading guideline or reading line using the following icons:

- Move the reading guideline upward or magnify the reading line

- Move the reading guideline downward or reduce the reading line

- Switching between horizontal/vertical reading guideline/line

- Switching between "red reading guideline" or "reading line"

Alternatively, you can use the **15** and **16** arrow keys to change the position of the „reading line“ or „reading row“.

### Adjusting maximum magnification

Select the icon for the maximum magnification factor that should be available.

### Adjusting false colors

Select this icon and then press the  button **3** to select the false color combination that should be available. Black/white plus inverting are activated by default. Non-activated combinations are shown crossed out.

Select the required combinations and then change their status using the  button.

### Adjusting the reading material brightness

Select this icon and then press the  button **3** to set the brightness for the reading material using the following icons:

- no LED lighting

- the LEDs light up with half power

- the LEDs light up with full power

- turn the LED spots on the screen on or off

Confirm the selected lighting with the  button.

## Adjusting the screen display brightness

EN

Select this icon and then press the  button **3** to adjust the display brightness using the following icons:

- Display brightness = 25%

- Display brightness = 50%

- Display brightness = 75%

- Display brightness = 100%

## Enabling/disabling the photo button

Select this icon to enable or disable the photo function of the  button **3**. If the photo button is disabled, the icon is crossed out. If you would like to enable the  button, press the  button again.

## Enabling/disabling the overview display

Select this icon to deactivate the overview display function. If you would like to enable the overview display again, press the  button **3** again.

## Changing the timeout settings

As standard, the device switches off automatically after approx. 15 minutes if it is not used. Select this icon and press the  button **3** to extend this period to approximately 8 hours.

## Updating the firmware

Select this icon and then press the  button **3** to display the current version of the firmware on the device and to update it, if necessary. If you have registered your vario DIGITAL 22 FHD, you will be notified about any available firmware updates. To update the firmware please proceed as follows:

1. Connect the vario DIGITAL 22 FHD to a PC using the provided USB cable. Ensure that the SD card is inserted in the card slot ⑪.
2. Turn the vario DIGITAL 22 FHD on. The contents of the SD card will now be displayed on the PC as the “ESCHENBACH” drive.
3. Now open the web address that was provided to you in the email about the firmware update using the connected PC.
4. Download the firmware update and save it directly on the drive - not in a drive subfolder.
5. Disconnect the vario DIGITAL 22 FHD from the PC using the “Unplug and eject hardware” function, whose icon you can find on your operating system taskbar.
6. Open the settings menu, select the  icon, and press the  button.
7. Select the  icon and press the  button. If you want to cancel the update and return to the previous page, select the  icon and press the  button.

The following icon will be displayed on the screen during the firmware update:

The vario DIGITAL 22 FHD will turn off automatically once the update is successful.

**Never turn the device off and never remove the SD card from the device during the firmware update.** Otherwise, the vario DIGITAL 22 FHD could be irreparably damaged.

This type of damage is not covered by the warranty!

You can use the  icon and confirm with the  button to quit the settings menu.

## USB port

You can use the supplied USB cable to connect the device from its USB connection ⑫ to the USB connection on another device, e.g. a computer. The SD card inserted in the device will then be displayed on the computer as the „ESCHENBACH“ drive for file operations.

## Reset

If the device no longer functions as described in these instructions, it may be due to a malfunction, which you can eliminate yourself by resetting the device. Then press the RESET button ⑬.

## Technical specifications

Display size:	54.6 cm (21.5")   16:9   Full HD
EN Camera:	optical/digital zoom camera with AF
Magnification range:	1,6 × – 100 ×
Display modes:	real color, contrasting up to 14 false color combinations possible
Card formats supported:	SD, SDHC (max. 32 GB)
Charger power supply:	Primary: 100 V – 240 V ~   50 – 60 Hz Secondary: 19 V ≡   2000 mA
USB port:	Type C (only for data transfer)
Operating temperature range:	+10 °C – +40 °C (50 °F – 104 °F), humidity: < 80 %
Storage temperature range:	0 °C – +60 °C (32 °F – 140 °F), humidity: < 80 %
Dimensions (W × H × D):	520 × 150 × 500 mm (20.5" x 5.9" x 19.7")
Weight:	~ 8.9 kg (19.6 lbs)
Lighting:	LED
Power consumption:	~ 30 W
Images per second.:	60 fps
Contrast ratio:	1000 : 1

These instructions are also available on the Internet at <http://www.eschenbach-optik.com/>

USB® is a registered trademark of USB Implementers Forum, Inc.

## Care instructions

- ▶ Turn off the device and disconnect it from the power source before you clean it.
- ▶ Clean the device using a soft, dry cloth only.
- ▶ Never use soap solutions containing plasticizers or alcoholic or organic solvents to clean your device, and never use abrasive detergents.
- ▶ Never clean the device in an ultrasonic bath or under running water.
- ▶ Protect your device against heavy contamination.
- ▶ Condensate may form and cloud the lens when you move from cold rooms to warm rooms. If this happens, wait 40 minutes before starting the device to avoid condensate formation.

## Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

## Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt.

# Supplier's Declaration of Conformity

## 47 CFR § 2.1077 Compliance Information

EN

### Unique Identifier

Trade Name: vario DIGITAL FHD

Model No.: 16592, 165922, 1659221, 1659222, 1659223

### Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

### FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ▶ Reorient or relocate the receiving antenna.
- ▶ Increase the separation between the equipment and receiver.
- ▶ Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ▶ Consult the dealer or an experienced radio/TV technician for help.

**MODIFICATION:** Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

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Eschenbach Optik GmbH