vario **DIGITAL** FHD Advanced

165916, 1659161, 1659162, 1659163

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Operating instructions **EN**

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Manual de instrucciones **ES**

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Bruksanvisning **SE**

Bruksanvisning **NO**

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Dear Customer,

Congratulations on the purchase of your Eschenbach vario DIGITAL FHD Advanced.

As market leader in magnifying glasses and reading aids we at Eschenbach can look back on many years of experience, which is reflected in the quality, handling and design of our products.

Firmware version

These instructions are based on firmware version "VD16ADV_v1.09" or higher.

Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images. To guarantee correct functioning, the screen reader must be set up as described.

The product is operated in a domestic environment (also includes nursing homes, workplaces, schools and other educational facilities).

Before first use

Please read these operating instructions thoroughly before using the device for the first time. This is the only way to get to know all the features available for the device.

Package contents

- ▶ vario DIGITAL FHD Advanced
 - ▶ Remote control with 2 × AAA batteries
 - ▶ Power adapter
 - ▶ Power plug adapter for EU, GB, US, AU
 - ▶ HDMI cable
 - ▶ USB cable (Type C)
 - ▶ Microfiber cloth
 - ▶ Operating instructions
- ▶ Depending on the version:
 - incl. battery
 - incl. X-Y table

Safety information

EN

- ▶ **Risk of damage to the eye and risk of injury: Never look into the light sources on the underside of the device.**
- ▶ **Never allow children to handle electric devices without supervision.**
- ▶ **Risk of suffocation and risk of accidental swallowing: Keep packaging, films and packaging parts away from children.**
- ▶ **Risk of pinching: ensure that your fingers do not get pinched during unfolding and folding of the device, and also during unfolding and folding of the camera on it.**
- ▶ **The entire device must stand on a stable surface.**
- ▶ **Take care not to leave any objects on the base when you want to fold the device together. This can result in damage to the display.**
- ▶ **Protect the device against impact or shock. Handle the device with care.**
- ▶ **Never subject the device to humidity or moisture.**
- ▶ **Protect the device from excessive heat and open flames. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in locations with excessive heat, for example in a parked car.**
- ▶ **Use only the supplied power adapter and country-specific power plug adapter.**
- ▶ **Check the device, the power adapter and its cable for visible damage. If there is any damage, disconnect the power adapter from the power supply immediately. Have the product checked by a qualified technician before you use it again.**
- ▶ **Class 1 laser in acc. w. EN 60825-1:2014.**
- ▶ **Route the power cord so that it does not present a tripping hazard.**
- ▶ **Never open the device housing. Opening the housing voids the warranty.**
- ▶ **The device must be turned off and the power plug must be removed before it is folded together.**

- ▶ **Carry the device only when it is folded together and by using the carrying handle.**
- ▶ **Only for domestic use.**
- ▶ **In a medical environment this device may not be operated together with other devices that are used for life support.**
- ▶ **Ensure that other people, especially children and persons who cannot assess this hazard, are aware of this hazard.**
- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**
- ▶ **If you purchased the device including batteries, they may only be removed or exchanged by authorized personnel.**
- ▶ **CAUTION risk of explosion! Never replace the battery with the wrong type. Never dispose of a battery in a fire. Never crush or cut a battery. Never leave a battery laying around in an environment with extremely high temperatures, as this can result in an explosion or cause flammable liquids or gases to escape. Never subject a battery to extremely low air pressures, as this can result in an explosion or cause flammable liquids or gases to escape.**
- ▶ **Dispose of used batteries in accordance with the legal requirements.**
- ▶ **When using the variant with X-Y table:**
 - **Make sure not to get your fingers caught when moving the table.**
 - **Only ever carry the device on the intended carrying handle when it is folded together. Fasten both brakes and press the button to lock the X-Y table (see page 48).**

Functional elements

(For illustrations see pages 2 to 4)

EN

1. Display
2. Rotary knob to open contrast functions
3. Photo button for saving images and confirming menu options
4. Button for turning the reading line on and off
5. On/Off button
6. Button for opening menu operation
7. Button for overview display
8. Rotary knob to increase or decrease zoom factor
9. USB port
10. SD card slot
11. SD card (insert card as shown)
12. Button for moving image selection upward
13. Button for moving image selection downward
14. Foldable camera cover (flipped open in illustration)
15. Camera
16. Button for moving image selection to the right
17. Button for moving image selection to the left
18. RESET button for resetting the device
19. Monitor arm for setting screen height and tilt angle
20. Base arm for unfolding the device
21. Charging control LED
22. Connection for power adapter
23. Carrying handle
24. Base plate
25. HDMI input
26. Rangefinder

Basic operation

Structure of the device and power supply

Only set up the appliance on a suitable surface, e.g. a table. Place the device on the base plate 24. Unfold the base arm 20 on the device until it engages (see illustration). The camera 15 is equipped with a cover 14. Open the camera cover and move it to the intended position (see illustration on pages 2 to 4). If the  symbol is shown, the camera cover has not been opened.

EN

Use the joint 19 to set the required screen height and tilt angle.

Connect the power adapter to a suitable device plug. Connect its cable to the connection for the power adapter 22. Then plug the power adapter into the power socket. The charging control LED 21 lights up while the device is being charged.

To fold the device together again move the screen backwards and downwards. Then fold the camera together and close the camera cover (see illustration). Thereafter, fold the base arm downward.

Turning on/off

Press the On/Off button 5. The camera image will appear on the screen 1 after a few seconds. Symbols briefly show the current status, for example magnification factor and brightness setting.

To turn the device off, press and hold the On/Off button 5 for one second. Current settings are saved.

Reading with the device

Turn the device on and place the reading material on the base plate  under the camera .

EN

In exceptional cases it can make sense to relocate the field of vision to the upper area of the base plate. To do this, swivel the camera out past the locking position. It is also possible to move the camera forwards, for example for carrying out handicrafts.

Adapting the magnification

To increase the magnification turn the rotary knob  in clockwise direction when the live image is displayed. To decrease the magnification, turn the rotary knob  counterclockwise. The magnification factor you have set will be displayed briefly.

Overview function

If you would like to see an overview of the reading material, then press the  button . The magnification will change to 1.3x. If you press the  button again, the previously set magnification will be selected again.

Dynamic Line Scrolling (DLS)

With the DLS function, you can move the image selection horizontally or vertically within the visible area without having to move the reading material.

To activate the DLS function, press and hold the  button  for approx. 3 seconds. The device then zooms to the preset width. That is the field of vision available for DLS. Now you can adapt the zoom factor by using the  knob.

With buttons , , , and  you can move the displayed section horizontally or vertically without having to move the reading material itself. The image selection can be moved upward with arrow key , downward with arrow key , to the right with arrow key , and to the left with the arrow key . If you press one of the arrow keys twice in rapid succession, the screen section is moved to the respective end. For example, you can quickly find the beginning of the next line by pressing the  arrow key twice.

To stop the DLS function, press and hold the  button . The  icon appears. Thereafter, the entire magnification range is available again.

There are three reading modes: Letter, Book und Column. You should select the mode which best matches the width of your reading material. The reading modes can be configured in the menu. The following reading modes are available, offering different magnification presets:

- ▶  Reading Mode 1: Letter (2.2 times – maximum 12 times magnification)
Use this reading mode for very wide objects (up to 215 mm/8.5")
- ▶  Reading Mode 2: Book (3.3 times – maximum 28 times magnification)
Select this mode to read a book (width based on reading mode) using DLS.
- ▶  Reading Mode 3: Column (6 times – maximum 48 times magnification)
To read (newspaper/magazine) columns using DLS, select this mode.

Alternatively, the DLS can also be operated using the remote control (see “Remote control” chapter on page 36).

Turning the reading line/row on and off

Press the **4** button to turn the reading line on. Press it again to hide the line.

Special button functions

- ▶ Press and hold the  button **3** to activate/deactivate DLS (see “Dynamic Line Scrolling (DLS)” chapter on page 34).
- ▶ Press and hold the  button **7** to turn the autofocus function off (see also Chapter “Deactivating/activating autofocus” on page 36).
- ▶ Press and hold the reading line button **4** to change the reading material brightness (see Chapter “Adjusting the reading material brightness” on page 42).

False color display

To display the camera image in false colors for increased contrast, turn rotary knob ② until the required false color rendering is displayed. By default, the false colors are preset as black/white and white/black. To adapt the false color display, open the “Adjusting false colors” menu (see Page 41).

Saving the live image as a file

Press the  button ③ briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the screen. You can also change the magnification factor for this image as already described. You can use the arrow keys ⑫ + ⑬ to move the magnified image section vertically, and arrow keys ⑯ + ⑰ to move it horizontally. Press the Photo button again to return to the live image display.

Deactivating/activating autofocus

Press and hold the  button ⑦ to deactivate the autofocus function. This can be helpful, for example while writing under the device, so that the focus remains on the reading material. Press the  button again to reactivate autofocus. Autofocus is automatically reactivated when the device is turned off.

Mirror function

If the camera is pointed toward the user and the screen is lowered slightly, it is possible for the user to look at himself. This can be useful, for example to apply make-up or for facial care.

Remote control

The remote control can be used while you are holding it in your hand and also when it is lying on the table. The anti-slip strips ensure that the remote control grips the table well. The remote control only works if there is a visual connection between yourself and the IR sensor below the display.

Setting-up for use

One battery terminal is insulated against the contact area with a plastic strip to avoid self-discharge. To remove it, open the battery compartment cover on the underside of the remote control and pull the strip out.

Adapting the zoom

To zoom in, press **A**. To zoom out, press **B**. The currently selected zoom factor is shown briefly in the display.

Dynamic Line Scrolling

To use Dynamic Line Scrolling, press and hold the OK button **F** briefly. Once the desired zoom factor has been set, you can use the arrows to move the displayed section horizontally or vertically without having to move the reading material itself. If you press the arrow buttons at the end/beginning of the image selection for 2 seconds, the image selection jumps to the other end/beginning. This makes it much easier to read text.

Overview function

If you would like to get an overview of the reading material, press  button **C**. The magnification changes to 1.3 x. By pressing the  button **C** again, the previously set magnification is selected again.

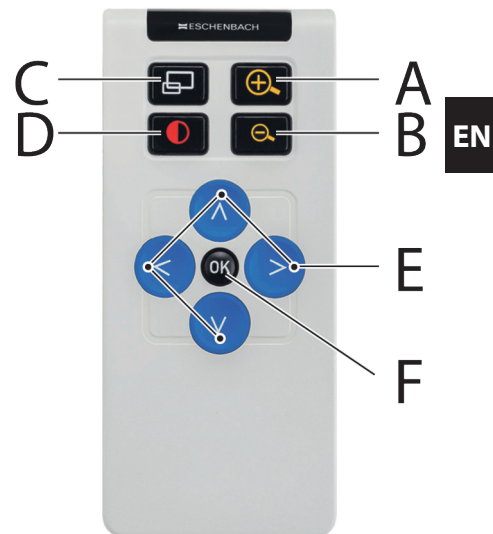
False color display

To change false colors, press the **D** button until the desired false color is displayed.

Channels

A switch for switching over the channels is located in the battery housing. However, this is only necessary if two devices are being used next to each other. You can set the relevant channel in the menu under "Remote control" (see page 43).

The channel selected on the remote control and in the menu must be identical. Only

then can the remote control be used. Channel 1 is used by default.

Opening the main menu

EN

Press the MENU button **6** to open the menu:

You can move to the individual symbols using the rotary knobs **2** + **8** and the arrow keys **12**, **13**, **16**, **17**. The symbol for the selected function becomes larger and is displayed in colour. Press the  button **3** to activate each selected function.

Individual menu commands are also played back acoustically. Playback can also be deactivated by changing the volume. To do this, press the  symbol several times until the  symbol is displayed.

If you press the  button **7**, the currently selected menu item is audibly played back again.

You can exit the setting menu at any time by simply pressing the MENU button **6**.

Opening the quick guide

Select the national flag that represents the language you desire. Press the  button to change the language. You can use  /  to scroll the page preview backward and forward. Operation is analog to viewing images from the SD card. Press the  button **3** to quit full-screen mode.

Managing image files on the SD card

Highlight this symbol and then press the  button to open image files from the SD card. The following menu appears:

The currently selected image file will be displayed centrally above it. The previous image is shown to the left of it, the next one to the right.

Use the two rotary knobs **2** + **8** to select the individual symbols with the following functions:

-  - select the previous image
-  - select the next image
-  - show the image file as full screen image
-  - delete the selected image from the SD card
-  - back one level

You can use the rotary knob **8** to set the required zoom factor. You can use the left-hand arrow key **16** + **17** to move the zoomed-in image section horizontally and the right-hand arrow key **12** + **13** to move it vertically.

To exit the full screen image, press the  button **3**.

Deleting image files

To delete the selected image file from the SD card, first select the delete symbol  and then confirm with the  button **3**. You can use the rotary knobs **2** + **8** to select  or . The symbol you have selected will be displayed slightly larger. Press the  button to execute the selected function.

Adjusting volume

The current volume setting is displayed. Select this symbol and then press the  button **3** to select the following settings:

- very low volume

- low volume

- medium volume

- high volume

- mute

Selecting the language for voice output

Select this symbol and then press the  button **3** to select the language for voice output. The symbol for the current voice output setting is displayed. Alternatively you can select  or  for confirmation using a signal tone.

The  symbol is displayed while the language file is being loaded.

Setting the date and time

The time will only be displayed in the menu once it has been set. Select this symbol and then press the  button **3** to set the time. Select the  symbol so that you can increase the number of the parameter displayed in yellow using the  button **3**. Select the  symbol so that you can decrease the number of the parameter displayed in white using the  button **3**. Select the  symbol and then press the  button **3** to select the next parameter to be set.

Select "24H" and press the  button **3** to change from a 24-hour display to the 12-hour display normally used in English-speaking areas or vice versa.

The date format normally used in the EU is set as standard. Select EU and use the  / 

symbols to change the date format if required.

Adjusting the reading line/row

Select this symbol and then press the  button **3** to set the details for the reading line or row using the following symbols:

- move the reading line position upward or magnify the reading row

- move the reading line position downward or minimize the reading row

- switch to "red reading line" or "reading row"

- switch between horizontal/vertical reading line/row

Alternatively you can use the **12** and **13** arrow keys to change the position of the "reading line" or "reading row".

Adjusting maximum magnification

Select the symbol for the maximum magnification factor that should be available. The selected zoom factor is ignored in DLS mode.

By default, the magnification is not saved after switching off. If you would like the magnification set last to be displayed, then highlight this  symbol and press the  button **3**.

- The magnification set last is not saved

- The magnification set last is saved

Adjusting false colors

Select this symbol and then press the  button **3** to select the false color combination that should be available. Black/white plus inverting are activated by default. Non-activated combinations are shown crossed out.

Select the required combinations and then change their status using the  button **3**.

Adjusting the reading material brightness

Select this symbol and then press the button ③ to set the brightness for the reading material using the following symbols:

- no LED lighting

- the LEDs light up with half power

- the LEDs light up with full power

- turn the LED spots on the screen on or off

Confirm the required brightness using the button ③.

Adjusting the screen display brightness

Select this symbol and then press the button ③ to adjust the display brightness using the following symbols:

- display brightness = 25 %

- display brightness = 50 %

- display brightness = 75 %

- display brightness = 100 %

Selecting the reading mode

Select this icon to set the reading modes described in the “Dynamic Line Scrolling (DLS)” chapter.

Opening the second menu level

Select this icon and press the button ③ to enter the second menu level.


Quit the menu

Select this icon and press the  button **3** to return to the live image display.

Second menu level

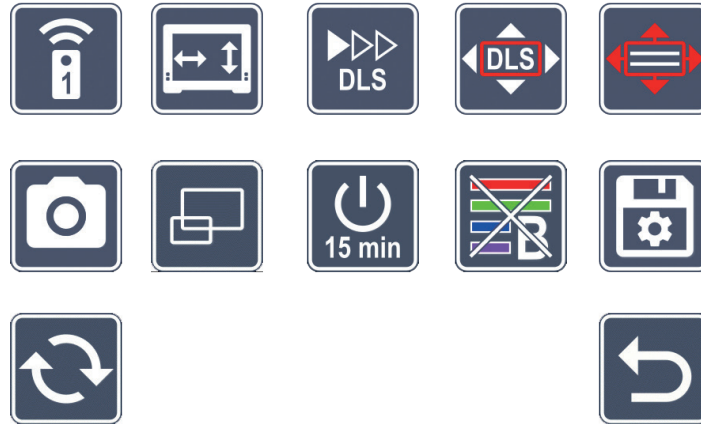The second menu level provides the following functions:

Remote control

Controlling two devices remotely in the same room may cause them to interfere with each other. This menu allows the user to set different communication channels for the remote control to prevent interference.

The channel selected on the remote control and in the menu must be identical. Only then can the remote control be used. Channel 1 is used by default.

Swapping navigation keys

Select this icon and press the  button **3** to switch the functionality of the lateral key pairs for moving the image selection. Horizontal movement is then carried out using the right-hand button pair, vertical movement using the left-hand button pair.

Setting the reading speed

EN

Select this icon and press the  button ③ to set the scroll speed at which the camera image moves across the display when using the image scrolling buttons:

- low scroll speed

- medium scroll speed

- high scroll speed

Enabling/disabling DLS

Select this icon and press the  button ③ to enable the Dynamic Line Scrolling (DLS) function. If the function is disabled, the symbol is crossed out. If you would like to enable the function, press the  button again.

Enabling/disabling vertical DLS

Select this icon and press the  button ③ to disable vertical Dynamic Line Scrolling (DLS). If this icon  is shown, only horizontal DLS is available.

Enabling/disabling the photo button

Select this icon to enable or disable the photo function of the  button ③. If the photo button is disabled, the symbol is crossed out. If you would like to enable the  button, press the  button again.

Enabling/disabling the overview display

Select this icon to disable the overview display function. If you would like to enable the overview display again, press the  button ③ again.

Changing the timeout settings

As standard, the device switches off automatically after approx. 15 minutes if it is not used. Select this icon and press the  button **3** to extend this period to approximately 8 hours.

Enabling/disabling the blue filter

The blue filter is disabled as standard. Select this icon and press the  button **3** if you want to enable it.

If the blue light filter is enabled, the amount of blue light is reduced. The display becomes more yellow and the colors appear softer.

Saving settings, loading and restoring factory settings

Select this icon and press the  button **3** to manage menu settings such as language, volume, activated false colors, LED settings, etc.:

- The first function allows you to save your current settings.

- This function loads the settings you saved previously. The requirement for this is that settings were saved previously.

Tip: Once you have set your device up for the first time you can restore your personal configuration using these two functions.

- This function allows you to reset the device to the default settings if required. All settings will then be as set at the default settings.

Updating the firmware

Select this symbol and then press the  button **3** to display the current version of the firmware on the device and to update it if necessary. The current firmware version can be found under:

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To update the firmware please proceed as follows:

1. Connect the vario DIGITAL FHD Advanced to a PC using the provided USB cable. Ensure that the SD card is inserted in the card slot ⑩.
2. Turn the vario DIGITAL FHD Advanced on. The contents of the SD card will now be displayed on the PC as the "ESCHENBACH" drive.
3. Download the firmware update and save it directly on the drive - not in a drive subfolder.
4. Disconnect the vario DIGITAL FHD Advanced from the PC using the "Unplug and eject hardware" function, whose symbol you can find on your operating system taskbar.
5. Open the settings menu, select the  symbol and press the  button.
6. Select the  symbol and press the  button. To cancel the update and return to the previous page, select the  symbol and press the  button.

The following icon will be displayed on the screen during the firmware update:

The vario DIGITAL FHD Advanced will turn off automatically once the update is successful. **Never turn the device off and never remove the SD card from the device during the firmware update.** Otherwise the vario DIGITAL FHD Advanced could be irreparably damaged. This type of damage is not covered by the warranty!

You can use the  symbol and confirm it with the  button ③ to quit this menu level and return to the main menu.

Connections

USB port

You can use the supplied USB cable to connect the device to the USB port on a computer using the device USB port ⑨. The SD card inserted in the device will then be displayed on the computer as the “ESCHENBACH” drive for file operations.

HDMI

You can use the enclosed HDMI cable to connect the device via its HDMI input ⑳ to the HDMI output of other devices. If the device detects the HDMI signal at the input, it automatically switches to the display of this signal. To switch back to the camera display, press and hold the MENU button ⑥ for 3 seconds. To switch back to the HDMI signal, press and hold the MENU button ⑥ for 3 seconds again. If the connection to the HDMI device is interrupted, the device automatically switches to the camera display.

This function depends on the output device. The camcorder or the camera must be fitted with an HDMI output and HDMI Live.

If you do not need sound, you can set this in the menu. Open the menu and set the volume (🔊) to muted (🔇).

Reset

If the device no longer functions as described in these instructions, it may be due to a malfunction, which you can eliminate yourself by resetting the device. Then press the RESET button ⑱.

Additional equipment variants

EN

Battery

If you own the vario DIGITAL FHD Advanced with a battery, please read this paragraph carefully:

- ▶ The battery may only be removed or replaced by the manufacturer or its customer service centre.
- ▶ It takes 4.5 hours to completely charge the battery, and this allows an operational duration of 5 hours.

X-Y table

If you own the vario DIGITAL FHD Advanced with X-Y table, please read this paragraph carefully:

27. Horizontal brake (unbraked)

28. Vertical brake (unbraked)

29. Button for locking the X-Y table

30. Button for releasing the lock

Before you can use the X-Y function, press the button to release the lock **30**. Then, move the brakes **27** and **28** to the shown position. You can now move the X-Y table horizontally

and vertically.

If you only move brake 27 right, the X-Y table can only be moved vertically. If you only move brake 28 up, the X-Y table can only be moved horizontally. For transport, only ever carry the device on the intended carrying handle when it is folded together. Fasten both brakes 27 and 28 and press the button to lock the X-Y table 29.

Technical specifications

Display size:	39.6 cm (15.6") 16:9 Full HD	
Camera:	optical/digital zoom camera with AF	
Magnification range:	1.3 × - 45 ×	
Display modes:	real color, contrasting up to 14 false color combinations possible	
Card formats supported:	SD, SDHC (max. 32 GB)	
Charger power supply:	Primary: 100 V – 240 V ~ 50 / 60 Hz Secondary: 19 V == 3420 mA	
USB port:	Type C (only for data transfer)	
HDMI input:	HDMI 1.4, depending on output device	
Operating temperature range:	+10 °C – +40 °C (50 °F – 104 °F), humidity: < 80 %	
Storage temperature range:	0 °C – +60 °C (32 °F – 140 °F), humidity: < 80 %	
Dimensions:	378 × 460 × 132 mm (14.9" x 18.1" x 1.6") (W × H × D)	
Dimensions with X-Y table:	435 x 500 x 160 (W x H x D)	
Weight:	~ 6.3 kg	~ 6.6 kg (incl. battery)
	~ 8.7 kg (incl. X-Y table)	~ 9.0 kg (incl. X-Y table and battery)
Lighting:	LED	
Power consumption:	~ 16 W	
Images per second.:	60 fps	
Contrast ratio:	1000 : 1	

These instructions are also available on the Internet at <http://www.eschenbach-optik.com/>

USB® is a registered trademark of USB Implementers Forum, Inc.

Care instructions

EN

- ▶ Turn off the device and disconnect it from the power source before you clean it.
- ▶ Clean the device using a soft, dry cloth only.
- ▶ Never use soap solutions containing plasticizers or alcoholic or organic solvents to clean your device, and never use abrasive detergents.
- ▶ Never clean the device in an ultrasonic bath or under running water.
- ▶ Protect your device against heavy contamination.
- ▶ Condensate may form and cloud the lens when you move from cold rooms to warm rooms. If this happens, wait 40 minutes before starting the device to avoid condensate formation.

Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt.

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Trade Name: vario DIGITAL FHD

Model No.: 16591, 165916, 1659161, 1659162, 1659163

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ▶ Reorient or relocate the receiving antenna.
- ▶ Increase the separation between the equipment and receiver.
- ▶ Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ▶ Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

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Eschenbach Optik GmbH