miniframe2

miniframe2 SUN

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Bruksanvisning

Käyttöopas

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 **ESCHENBACH**

Deutsch	3
English	5
Français	7
Italiano	9
Español	11
Nederlands	13
Dansk	15
Svenska	17
Norsk	19
Suomi	21
Polski	23
Česky	25
日本語	27

Deutsch

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English

**Important safety instructions,
please read carefully!**

You have purchased a quality product from the Eschenbach company.

Intended use

This vision aid is intended for short term use as replacement or supplemental eyeglasses. It is designed not only for the immediate reading distance but also for an expanded close-up distance. The optical value of the lenses decreases from the bottom to the top.

Warning!

- Only intended for close-up vision and reading.
- Regular professional eye examinations are essential for optimal vision correction and evaluation of the health of your eyes.
- Do not use while driving automobiles.
- Not suitable for seeing at a distance.
- Do not use as an eye protection device.
- Not suitable for looking directly into the sun.
- The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

Only miniframe2 sun:

- Filter category 2 as per DIN EN ISO 12312-1 in its currently applicable version.

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Eschenbach Optik GmbH