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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Bruksanvisning

Käyttöohje

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DE

EN

FR

IT

ES

NL

DA

SV

NO

FI

PL

CS

JA

prismatic BINO Comfort

Deutsch.....	3
English	4
Français.....	5
Italiano	6
Español	7
Nederlands.....	8
Dansk	10
Svenska	11
Norsk.....	12
Suomi	13
Polski	14
Česky.....	15
Русский.....	16

Deutsch

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English

Congratulations on your purchase of the prismatic BINO Comfort prismatic half moon glasses from Eschenbach. The glasses have been calculated for the best optical and anatomical use situation. Special emphasis was placed on good wearing comfort.

Intended use

This product is designed exclusively for providing enlarged images of text, pictures, or other object details.

Important safety instructions, please read them attentively!

- ▶ **Risk of fire!** Lenses in optical devices can cause significant damage through the "burning-glass effect" if handled or stored improperly. Take care to never to leave optical lenses lying in direct sunlight.
- ▶ **Risks of glare and injury!** Never look directly into the sun or other bright light sources when using optical devices.
- ▶ **Also inform other people and children especially of this safety measure!**
- ▶ **Protect your prismatic half moon glasses against impacts or shocks and excess heat.** Never place your prismatic half moon glasses on radiators or in direct sunlight.
- ▶ **These prismatic half moon glasses are suitable only for reading and may not be used in road traffic.**
- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Utilisation

1. Use appropriate lighting, one which allows you to evenly illuminate your reading material and does not cause reflections.
2. Put your prismatic half moon glasses on.
3. To find the correct reading distance, hold the reading material close to your glasses.
4. Slowly move the reading material away from the glasses until the print is seen as a sharp image. This is the correct reading distance.
 - The print appears blurred when the reading distance is too large or too small.

Reading distances for different prismatic half moon glasses

- The reading distance is dependent on the magnification of the reading glasses being used.

Article number	Diopters	Reading distance
168014 / 168054	4 dpt	approx. 250 mm
168015 / 168055	5 dpt	approx. 200 mm

Article number	Diopters	Reading distance
168016 / 168056	6 dpt	approx. 165 mm
168018 / 168058	8 dpt	approx. 125 mm
1680110 / 1680510	10 dpt	approx. 100 mm

If you have questions, please always contact your dealer.

Care Instructions

- ▶ **Clean the lenses with a soft cotton or linen cloth (such as eyeglass cleaning cloth)**
- ▶ **For heavy soiling (e.g. fingerprints), clean the lenses with lukewarm water and dry them.**

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH