ambelis

Sicherheits- und Pflegehinweise
Safety and care instructions
Consignes de sécurité et d'entretien
Indicazioni di sicurezza e manutenzione
Indicaciones de seguridad y de cuidado
Veiligheids- en onderhoudsinstructies
Sikkerheds- og plejeanvisninger
Säkerhetsanvisningar
Sikkerhets- og vedlikeholdshenvisninger
Turva- ja hoito-ohjeet
Wskazówki dotyczące bezpieczeństwa i czyszczenia
Bezpečnostní pokyny a instrukce k péči
Руководство по эксплуатации
安全上／お手入れ時のご注意

 **ESCHENBACH**

Deutsch

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English

Intended use

ambelis products help protect your eyes from harmful UV light and improve contrast perception. The filter type/category depends on the environment and glare sensitivity.

Safety instructions

- ▶ **Danger of blinding or injury!** This product does not protect your eyes when looking directly at the sun - it is not suitable for the direct look at the sun!
- ▶ The product does not protect against artificial light sources, such as used in tanning shops.

- ▶ The product must not be used as eye protection against mechanical hazards such as bumps, flying splinters etc..
- ▶ Protect your ambelis products from knocks, strikes, moisture and excessive heat! Never place them on a heating element or in sunlight!
- ▶ Ensure that other people, especially children, are aware of these dangers!

Care instructions

- ▶ Do not use any soap solutions that contain softeners, alcohol-based or organic solvents or abrasive cleaning products! These may damage the lenses.

Clean your ambelis products with a soft, lint-free cloth (e.g. spectacle cleaning cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. fingerprints).

ambelis complies with standard DIN EN ISO 12312-1 as well as regulation (EU) 2016/425 in their currently valid version.

Filtercategory of ambelis

ambelis filter type 'Classic':

Product (part.-no.)

filter category

ambelis 15

1

ambelis 65

2

ambelis 85

3

ambelis 50-15

2

ambelis 75P, polarized

2

ambelis active, phototropic

1/3

9

ambelis filter type 'Drive':

Product (part.-no.)

filter category

ambelis 15 Drive

1

ambelis 65 Drive

2

ambelis 85 Drive

3

ambelis 50-15 Drive

2

ambelis 75P Drive, polarized

2

10

 = not suitable for driving motor vehicles

 = Roadworthy by day, but not suitable for driving at night or in conditions with reduced brightness

 = very limited attenuation of solar radiation

 = limited attenuation of solar radiation

 = good protection against solar radiation

 = high protection against solar radiation

 = very high level of protection against extreme solar radiation, such as by the sea, on snow, in high mountains or in deserts

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Eschenbach Optik GmbH