visolux **DIGITAL** XL FHD

165511

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Operating instructions EN

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Manual de instrucciones ES

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Bruksanvisning SE

Bruksanvisning NO

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ESCHENBACH

Deutsch	1
English	22
Français	44
Italiano	64
Español	85
Nederlands	105
Dansk	125
Svenska	146
Norsk	166
Suomi	186
Polski	206
Česky	227
日本語	248

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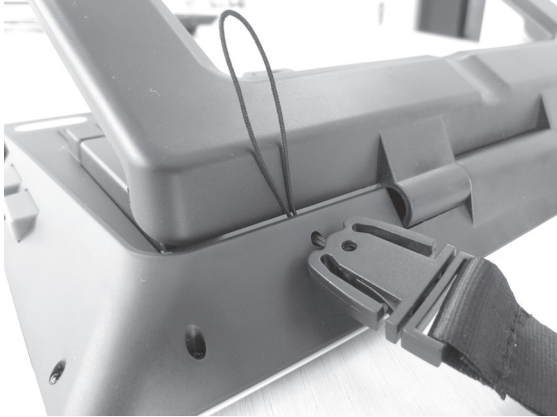
Dear Customer,

EN

we would like to congratulate you on purchasing the Eschenbach visolux DIGITAL XL FHD.

As the market leader for magnifying glasses and reading aids, we at Eschenbach can look back over many years of experience and this is reflected in the quality, usability and design of our products.

Firmware version

This manual is based on firmware version “VD12_fw_1.0.0” or higher. The current firmware version can be found under:

www.eschenbach-vision.com/de-DE/produkte/produkt/visolux-digital-xl-fhd16551

Intended use

This electronic reader is intended exclusively for enlarging and reading or viewing texts or images.

Before first use

Please read this manual carefully through before first use. This is the only way to get to know all the functions on the device.

Package contents

- ▶ visolux DIGITAL XL FHD
- ▶ Rigid foam case
- ▶ Carry strap
- ▶ Microfibre cloth
- ▶ HDMI cable
- ▶ Magnetic USB cable (Type-C)
- ▶ Universal charging device with adapters for EU, GB, US, AU
- ▶ Operating instructions

Safety information

- ▶ **Risk of glare and injury: Do not look into the light sources on the underside of the housing.**
- ▶ **Do not allow children to handle electronic devices without supervision.**
- ▶ **Only charge the reader with the mains adapter supplied.**
- ▶ **Choking hazard and danger of swallowing: Keep packaging, foils and packaging parts away from children.**
- ▶ **Check the device and the mains adapter with power cable for visible damage. If there is any damage, disconnect the connected power unit from the power supply immediately. Ask a qualified specialist to check the product before you use it again.**
- ▶ **Lay the mains cord so that nobody can trip over it.**
- ▶ **Never open the device housing. There are no serviceable parts whatsoever inside. Opening the device will void the guarantee.**
- ▶ **The fitted lithium-polymer batteries may only be removed or exchanged by authorised personnel.**
- ▶ **Caution: Risk of explosion when the wrong type of batteries is used. Dispose of used batteries in accordance with the requirements.**
- ▶ **Never expose the device to moisture.**
- ▶ **The light source incorporated in this lamp must only be replaced by the manufacturer, its Customer Services or another qualified person.**
- ▶ **Risk of trapping: Be careful not to trap your fingers when folding in the stand.**
- ▶ **Protect the device from impact or shock and excessive heat. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in places with excessive heat, such as in a parked car.**
- ▶ **Only intended for use in a domestic environment.**
- ▶ **In a medical environment, this device must not be operated with other life support equipment.**

- ▶ **Before using, check that the charging cable magnet is intact. If the magnet is damaged, fragments can be swallowed, and splinters can cause cuts**
- ▶ **Ensure that the magnets are kept out of the reach of children.**
- ▶ **Magnets can affect the function of pacemakers and implanted defibrillators. If you wear such devices, keep a sufficient distance (at least 6"/15 cm) from magnets. Warn wearers of such devices against approaching magnets.**
- ▶ **Keep magnets away from all devices and objects that can be damaged by strong magnetic fields, e.g. pacemakers, laptops, computer hard drives, credit and debit cards, data carriers, hearing aids.**
- ▶ **Some people have an allergic reaction to contact with nickel. Avoid prolonged skin contact with magnets. Avoid handling magnets if you already have a nickel allergy.**
- ▶ **Make other people - especially children - aware of these dangers.**
- ▶ **The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. If the device is used outside the EU member states, the respective country-specific directives must be observed.**

Operating elements

(For figures, see fold-out pages A1 and A2)

EN

1. Display
2. Contrast button for calling up contrast functions
3. Photo button for saving images and confirming menu options
4. On/Off button
5. Button for decreasing zoom factor
6. Button for increasing zoom factor
7. HDMI input
8. Button for moving image selection to right
9. Button for moving image selection to left
10. Cover over SD card slot
11. SD card (insert with contacts in front facing upwards)
12. Button for opening menu control
13. USB / charging connection
14. Charging check LED
15. Foldable stand foot
16. Camera (movable for reading or writing)
17. HDMI output
18. Button for moving image selection upwards
19. Button for moving image selection downwards
20. Reset key for resetting the device (on underside)

Basic operation

Before you can use the visolux DIGITAL XL FHD for the first time, you must fully charge the internal battery.

To do this, fit the appropriate country adapter to the power supply. Then plug the power supply into the mains plug and connect the cable to the USB connection at the power supply. A magnetic USB-C charging cable is included in the package contents. The magnetic force greatly simplifies plugging in and unplugging the device. To make use of this force, insert the magnetic connection into the USB connection ⑬. The charging cable can now be connected with the magnet.

The charging indicator LED ⑭ illuminates red while the device is being supplied with power. When the device is fully charged, the charging indicator LED ⑭ illuminates green.

You can use the device while it is charging, and it can remain permanently plugged in.

Switching on/off

Press the on/off button ④ to switch the device on. After a few seconds the camera image will appear on the display ①. Symbols briefly show the current status, e. g. regarding battery charging, zoom factor and brightness setting.

Press and hold the On/Off button ④ for one second to switch the device off. The current settings are saved.

Reading with the device

Unfold the stand ⑮. Switch the device on and place it on the reading material.

If you are using the device with the minimum zoom factor 2× you will see the complete image section approx. 12 cm wide on the display. If the zoom is increased, the image section will decrease.

On the rear you can relocate the camera position:

Position at bottom = reading mode, position at top = writing mode.

Dynamic Line Scrolling (DLS)

Thanks to the scrolling function it is not necessary to move the device to one side in order to read a complete newspaper column, for example, even at high zoom factors.

To move the image selection to the right, press the scroll button ⑧ (repeated pressing or keep pressed down).

To move the image selection to the left, press the scroll button ⑨ (repeated pressing or keep pressed down).

Press the scroll button ⑧ twice in quick succession to jump back to the right-hand limit stop. Press the scroll button ⑨ twice in quick succession to jump back to the left-hand limit stop. This enables you to find the beginning of the line quickly. The beginning and end are indicated by a red line at the edge of the screen.

The scroll facility is also available vertically in the same manner using the scroll buttons ⑱ + ⑲.

Live touchscreen

You can move the camera image directly on the display. To do this, tap one of your fingers on any point in this display and then wipe your finger across the display.

All scroll functions are only available at zoom factors greater than twofold.

Tip:

You will find noticeable bulges on the front edge which mark the side-ward limits of the visible area.

The stand foot ⑮ is provided with a notch (see figure) which is arranged in alignment with the camera axis. You can use this notch to place the device over round objects more easily, for example to read ingredient information on tins, etc.

Adapting the zoom

To increase the zoom factor, press and hold the ⊕— button ⑥ when the live image is being displayed. To decrease the zoom factor, press and hold the ⊖— button ⑤. Press the buttons briefly and repeatedly to change the zoom factor in steps.

You can also place two fingers on the display and move them away from each other while still touching the display in order to increase the zoom factor. You can also place two fingers on the display and move them towards each other to decrease the zoom factor.

The zoom factor currently being displayed is displayed briefly at the top left.

Overview function

With this function you can gain an overview of the reading material. Press the ⊕— button ⑥ and the ⊖— button ⑤ simultaneously to set the lowest magnification (twofold). Press the ⊕— button ⑥ and the ⊖— button ⑤ again to reselect the previously set magnification. The respectively set zoom factor is displayed briefly when the button is released.

Special functions of the buttons

- ▶ Keep the photo button ③ depressed to open the photo album.
- ▶ Keep the contrast button ② depressed to adapt the reading material illumination.

False colour display

In order to display the camera image in false colours for contrast amplification, press the contrast button ②. Press this button again, if necessary repeatedly, to switch through the various false colour modes and to return to the true colour display. The false colours are preset to black/white and white/black as delivered.

Call up the menu “Adapting false colours” to adapt the false colour displays.

Saving the live image as a file

Press the photo button ③ briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the display. You can also change the zoom factor in this image as described above. Briefly press the photo button again to return to the live image display.

Calling up images from the SD card

To display images previously saved on the SD card, press and hold the photo button ③ for 3 seconds or open the settings menu as described below using the MENU button ⑫ and then tapping the  symbol twice.

You can tap or use the two scroll buttons ⑧ + ⑨ to select the individual symbols which have the functions as described below:

-  - select the previous image
-  - select the next image
-  - show the current image as full screen image
-  - delete the selected image file from the SD card
-  - back one level

The symbol for the selected function is always shown slightly larger. To activate this function, tap the symbol or press the photo button ③.

To display the image file shown in the centre at full screen size, tap the image or press . Use the ⊕— and ⊖— ⑥ + ⑤ buttons or the finger movements described above to set the required zoom factor. You can move a zoomed-in image section horizontally across the display using the left-hand scroll buttons ⑧ + ⑨ and you can use the right-hand scroll buttons ⑱ + ⑲ to move it vertically across the display.

Tip: You can also relocate the zoomed image using your finger.

If the image is zoomed out you can use the top left-hand scroll button ⑧ or the bottom right-hand scroll button ⑲ to display the next image file at full screen size, and you can use the bottom left-hand scroll button ⑨ or the top right-hand scroll button ⑱ to display the previous image file.

If an image file is being displayed you can press the MENU button ⑫ to display the described 5 menu symbols again.

Deleting image files

In order to delete the image file shown at the centre of the screen from the SD card, tap the delete symbol . If you tap the delete symbol  again, the confirmation symbol  is shown on the left and the cancel symbol  on the right.

Tap  to delete the file. Tap  to cancel the delete process.

Tap  twice or press the MENU button  to return to the live image.

Settings

Press the MENU button  to call up the first level of the menu:

The menu offers a voice output. If you tap a symbol once, this designation is played back as voice output. Tapping the symbol again will activate it.

You can either move to the individual symbols using the scroll buttons or simply tap on them with your finger. The symbol for the selected function is always shown slightly larger and with a colour marking. To switch the selected function on or off, or to call up a saved submenu, tap the symbol again or press the photo button .

Tip: It is preferable to control the functions in the submenus by tapping them on the touch display as well. The buttons described above are also available there.

You can quit the menu at any time by pressing the MENU button .

Switching reading guideline/line on/off

Tap twice to switch the reading guideline/line on or off. If the reading aid function is switched on, the  symbol is displayed, if the function is switched off the  symbol is displayed.

Managing photo albums on the SD card

Tap twice to return to the submenu described in the previous chapter. If you have called up image file management via the settings menu, you can use the  symbol to return to the first level of the settings menu.

Calling up the quick guide

Tap twice and then tap the language symbol, if necessary repeatedly, in order to select the required language. You can use  /  to scroll forwards or backwards in the page preview. The operation is analogous to viewing images from the SC card. In order to quit full screen mode from a zoomed-out display, press the button  ⑥.

Adapting the zoom mode

Tap twice to switch the infinitely-variable zoom function on or off using the  ⑤ and  ⑥ buttons. If the function is switched on, the  symbol is displayed, if the function is switched off the  symbol is displayed.

Adjusting the volume

The current volume setting is displayed. Tap repeatedly to select the following settings:

- medium volume

- high volume

- muted

Adjusting the reading material illumination

Tap twice to adjust the illumination for the reading material using these symbols:

- switch the complete LED illumination on

- switch the complete LED illumination off

- only the bottom LED row is switched on (tap twice)

- the top LEDs right and left light up with half power (tap twice)

- the top LEDs right and left light up with full power (tap twice)

If you tap  twice you will return to the first level of the settings menu.

Tip: You can also adjust the illumination stages mentioned above for the top LEDs outside the menu by prolonged pressing and holding of the contrast button ②.

Adjusting display brightness

Tap twice and then twice on the symbol for the required display brightness:

- display brightness = 50 %

- display brightness = 75 %

- display brightness = 100 %

Adapting false colours

Tap twice and then twice on the false colour combinations which should be available later on when pressing the contrast button ②. Black and white plus inversion are activated ex-works. Non-activated combinations are shown crossed out.

Calling up the second menu level

Tap twice to call up the second menu level.

Quit the menu

Tap twice to return to the live image display.

EN

Extended settings

Adapting the reading guideline/line

Tap twice to set further details using these symbols:

- Tap to switch between 'red reading guideline' or 'reading line'

- Tap to show a thinner reading guideline

- Tap to show a thicker reading guideline

- Tap repeatedly to move the reading guideline upwards or to magnify the reading line

- Tap repeatedly to move the reading guideline downwards or to demagnify the reading line

Adjusting the touch screen

The symbol for the current touch screen setting is displayed. Tap repeatedly to select the following settings:

-  - Deactivate the touch function during live image display. Touch operation is retained in the menus.
-  - Diagonal image movement possible using live touchscreen operation
-  - Image movement only horizontally and vertically possible using live touchscreen operation (specially designed to simplify reading with DLS)

Setting the date and time

Tap twice to set the date and time. Tap the number you want to set. Repeatedly tap the  or  symbol to increase or decrease the parameter depicted in white. Tap the  symbol to select the next parameter for adjustment.

"24H" or "12H" stand for the 24- or 12-hour display:

24H - 24-hour display selected

12H - 12-hour display selected

The icon  takes you back to the settings menu.

Switching the photo button on/off

Tap twice to switch the photo button on and off.

-  - Photo button switched on
-  - Photo button switched off

Switching the overview display on/off

Tap twice to switch the overview display on and off.

- Overview display switched on

- Overview display switched off

Switching the dual screen function on/off

The symbol for the current dual screen function setting is displayed. This allows you to determine whether the built-in display will continue to be displayed if an HDMI device is connected. Tap the symbol repeatedly to select the following settings:

- dual screen function switched on

- dual screen function switched off

Switching the blue light filter on/off

By default, the blue light filter is deactivated. Tap  twice to switch on the blue light filter. The blue light portion is reduced. The display has a yellowish tint, and the colours appear softer and more mellow.

- Blue light filter switched on

- Blue light filter switched off

Changing the timeout settings

Ex-works, the device switches off automatically after approx. 5 minutes if it is not used. Tap twice on  to extend this period to approx. eight hours (during mains operation).

- Timeout after approx. 5 minutes

- Timeout after approx. 8 hours

Adapting the maximum zoom

Tap twice and then tap twice on the symbol for the zoom factor which should be available as a maximum.

Adapting reading speed

Tap twice to set the scrolling speed using these symbols, with which the live image runs over the display when using the scroll buttons:

- low reading speed

- medium reading speed

- high reading speed

If you tap these symbols twice, this setting will be applied, and an example of the reading speed will be shown in the background.

Selecting the language for language output

The symbol for the current language output setting is displayed. If necessary, tap the symbol repeatedly to select one of the available languages: Or select  or  for a signal tone confirmation.

While loading the language file, the  symbol appears.

Displaying the battery condition

Tap twice to show information on the battery condition.

- Current battery level is displayed

- Number of battery cycles are displayed

- Maximum capacity: This is a measurement for the battery performance which is relative to the performance of a new battery. A low capacity can possibly lead to a shorter period of use between charging cycles.

- Capacity too low: It is recommended to replace the battery as its capacity is too low.

Updating the firmware

Tap twice to display the current device firmware version and to update it if necessary. The current firmware version can be found under:

www.eschenbach-vision.com/de-DE/produkte/produkt/visolux-digital-xl-fhd16551

Tip: Have the firmware updated by your optician or your Eschenbach specialist dealer!

Never switch the device off and never remove the SD card from the device during a firmware update. Otherwise, the device could be irreparably damaged. This type of damage is not covered by the guarantee! Power the device from the mains using the mains adapter. This ensures that the update does not fail due to a lack of power supply.

To update the firmware please proceed as follows:

1. Connect the device USB connection ⑬ to a PC using the enclosed USB cable. Ensure that the SD card ⑪ is located in the device card slot.
2. The contents of the SD card will now be displayed on the PC as a drive.
3. Download the update, unpack the archive and save the "VD12FW.brn" file directly to the SD card.
4. Select the "Safely remove hardware and eject media" function on the PC - you can find the symbol for this function in the operating system taskbar. Then disconnect the connection to the PC.
5. To start the update, go to the "Update firmware" menu and tap  twice. If you would prefer to cancel the update and return to the previous page, tap  twice. During the update, a message appears on the display.

The device switches itself off automatically after a successful update.

Reset settings to factory settings

Tap twice to return all settings to the factory settings.

Quit the menu

EN Tap twice to return to the first level of the settings menu.

Connections

HDMI input

You can use the enclosed HDMI cable to connect the device to the HDMI output of other devices using its HDMI input ⑦.

When the device recognises the HDMI signal at the input, it automatically switches to display this signal. To switch back to the camera screen, press and hold the MENU button ⑫ for 3 seconds. To switch back to the HDMI signal, press and hold the MENU button ⑫ for 3 seconds.

If the connection to the HDMI device is interrupted, the device automatically switches back to the camera image.

HDMI output

You can use the enclosed HDMI cable to connect the device via its HDMI output ⑰ to the HDMI input of another device, e.g. a television. This enables both the camera image and the image files to be displayed on the connected device.

USB connection

You can use the enclosed USB cable to connect the device via its USB connection ⑬ to the USB connection of a computer. The SD card inserted in the device will then be made available to the computer as the "ESCHENBACH" drive for file operations.

In addition, the device can be supplied with power and charged via this connection (included power supply or USB power supply, for example from a computer).

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Trade Name: visolux DIGITAL XL FHD

Model No.: 165511

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ▶ Reorient or relocate the receiving antenna.
- ▶ Increase the separation between the equipment and receiver.
- ▶ Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ▶ Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

EN

UK Importer:

Associated Optical Products

13 Gore Road

SL18AA Burnham,Bucks

Internet: www.associatedoptical.com

Technical data

Display size:	29.5 cm (11.6") 16:9 Full-HD
Zoom range:	2× – 22×, also infinitely variable
Display modes:	Real colour, contrast-enhanced up to 14 false colour combinations selectable
Operating time:	approx. 3:00 hours
Charging time:	approx. 4:30 hours
Card formats supported:	SD, SDHC (max. 32 GB)
Power supply charging device:	100 V–240 V~ 50/60 Hz
Battery:	2x lithium-polymer
USB connection:	Type C
HDMI input/output:	HDMI 1.4, depending on input/output device
Temperature range:	+10 °C to +40 °C
Dimensions (W × H × D):	290 × 200 × 40 mm
Weight:	1050 g

This manual is also available on the Internet on:

<https://www.eschenbach-vision.com/de-DE/produkte/produkt/visolux-digital-xl-fhd16551>

USB® is a registered trademark of USB Implementers Forum, Inc.

HDMI and the HDMI logo are trademarks or registered trademarks of the HDMI Licensing LLC in the USA and other countries.

Reset

If the device stops working as described in this manual, it may be due a fault that you can rectify yourself by resetting the device. Then press the RESET button 20.

Care instructions

EN

- ▶ When cleaning the device do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents!
- ▶ Do not clean the device in an ultrasonic bath or under flowing water!
- ▶ Only use a soft, dry cloth to clean the device. For heavier soiling (e.g. fingerprints), moisten the cleaning cloth slightly.

Disposal

- ▶ This product is subject to the European WEEE directive on old electrical and electronic equipment as amended. Do not dispose of your old device with household waste, but via an authorised waste disposal company or your municipal waste disposal facility. Observe the regulations applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally responsible manner.

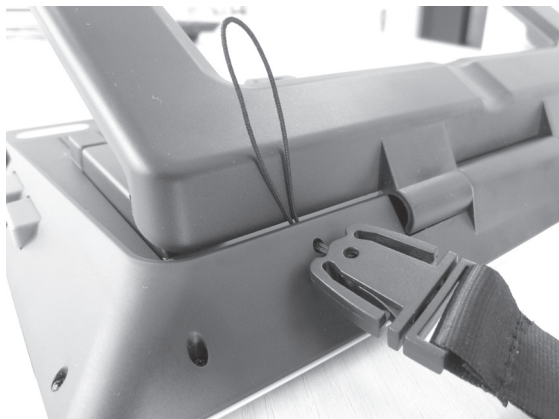
Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty only by proof of purchase.

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Eschenbach Optik GmbH