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Safety and care instructions

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English

About these instructions	9
Intended use	9
Safety instructions	9
Product description	10
Applicable standards	10
Care instructions	11
Disposal	11
Filter category	12
Explanation of symbols	12

About these instructions

These instructions contain important information for the safe use and proper cleaning of your product.

Therefore, read these instructions carefully and thoroughly before using this product.

Keep these instructions for future reference.

Intended use

The acunis product may only be used for protection against UV radiation from solar radiation. Intended use also includes observing these instructions.

Safety instructions

The acunis product was designed with great care and to be safe. Nevertheless, situations may arise during use that pose a risk to your health.

Important information for your safety

Read and follow the following safety instructions carefully! Make other people aware of these dangers.

- Risk of glare and injury! This product does not protect your eyes when looking directly into the sun.

- The product does not protect against artificial light sources, e.g. in solariums.
- The product must not be used as eye protection against mechanical hazards such as impacts, flying splinters, etc.
- Protect your acunis product from impact or shock, moisture and excessive heat! Never place it on radiators or in sunlight!

Product description

The acunis product protects your eyes from harmful UV light and improves contrast

perception. The filter variant/category depends on the environment of use and sensitivity to glare.

Applicable standards

Filter glasses⁽¹⁾ and filter clips⁽²⁾ for protecting the eyes from sunlight fulfil the standard DIN EN ISO 12312-1 as well as the regulations (EU) 2016/425⁽¹⁾ and 2017/745^(1, 2) in the currently valid version.

The applicable declaration of conformity can be found on the following website.

<https://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>

Alternatively via the following QR code:

If an older model is no longer listed here, please contact info@eschenbach-optik.com.

Care instructions

Clean your acunis product with a soft, lint-free cloth. For

heavier soiling, moisten the cleaning cloth slightly.

Avoid cleaning the product with:

- Soap solutions that contain plasticisers
- Alcoholic or organic solvents
- Abrasive cleaning agents

Failure to observe the instructions may result in damage to the product.

Disposal

If necessary, dispose of the product with household waste.

Filter category

Filter	Filter category
acunis 25 / ac 25	1  
acunis 50 / ac 50	1  
acunis 75 / ac 75	2  
acunis 75 SM (silver mirrored)/ ac 75 SM	3  

Explanation of symbols

Symbol	Meaning
	= Usable during the day, not suitable for driving at night or in conditions of reduced brightness.
	= Limited attenuation of solar radiation
	= Good protection against solar radiation
	= High protection against solar radiation

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Eschenbach Optik GmbH