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Safety and care instructions

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English

About these instructions	11
Intended use	11
Safety instructions	11
Product description	12
Applicable standards	12
Care instructions	13
Disposal	13
Filter category	14
Explanation of symbols	16

About these instructions

These instructions contain important information for the safe use and proper cleaning of your product.

Therefore, read these instructions carefully and thoroughly before using this product.

Keep these instructions for future reference.

Intended use

The ambelis product may only be used for protection against UV radiation from solar radiation. Intended use also

includes observing these instructions.

Safety instructions

The ambelis product was designed with great care and to be safe. Nevertheless, situations may arise during use that pose a risk to your health.

Important information for your safety

Read and follow the following safety instructions carefully! Make other people aware of these dangers.

- Risk of glare and injury! This product does not protect

your eyes when looking directly into the sun.

- The product does not protect against artificial light sources, e.g. in solariums.
- The product must not be used as eye protection against mechanical hazards such as impacts, flying splinters, etc.
- Protect your ambelis product from impact or shock, moisture and excessive heat! Never place it on radiators or in sunlight!

Product description

The ambelis product protects your eyes from harmful UV light and improves contrast perception. The filter variant/category depends on the environment of use and sensitivity to glare.

Applicable standards

Filter glasses⁽¹⁾ and filter clips⁽²⁾ for protecting the eyes from sunlight fulfil the standard DIN EN ISO 12312-1 as well as the regulations (EU) 2016/425⁽¹⁾ and 2017/745^(1, 2) in the currently valid version.

The applicable declaration of conformity can be found on the following website.

<https://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>

Alternatively via the following QR code:

If an older model is no longer listed here, please contact info@eschenbach-optik.com.

Care instructions

Clean your ambelis product with a soft, lint-free cloth. For heavier soiling, moisten the cleaning cloth slightly.

Avoid cleaning the product with:

- Soap solutions that contain plasticisers
- Alcoholic or organic solvents
- Abrasive cleaning agents

Failure to observe the instructions may result in damage to the product.

Disposal

If necessary, dispose of the product with household waste.

Filter category

ambelis filter variant Classic:

Filter	Filter category
ambelis 15 / am 15	1  
ambelis 65 / am 65	2  
ambelis 85 / am 85	3  
ambelis 50-15 / am 50-15	2  
ambelis 75P, polarising / am 75P	2  
ambelis active, self-tinting / am active	1 / 3   

ambelis filter variant Drive:

EN

Filter	Filter category	
ambelis 15 Drive / am 15D	1	 
ambelis 65 Drive / am 65D	2	 
ambelis 85 Drive / am 85D	3	 
ambelis 50-15 Drive / am 50-15D	2	 
ambelis 75P Drive, polarising / am 75P D	2	 

Explanation of symbols

Symbol	Meaning
	= Not for use on public roads
	= Usable during the day, not suitable for driving at night or in conditions of reduced brightness.
	= Very limited attenuation of solar radiation
	= Limited attenuation of solar radiation
	= Good protection against solar radiation
	= High protection against solar radiation

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Eschenbach Optik GmbH