# basic visionLED

**Bedienungsanleitung**  
**User Manual**  
**Mode d'emploi**  
**Istruzioni d'uso**  
**Manual de instrucciones**  
**Gebbruiksaanwijzing**  
**Betjeningsvejledning**  
**Bruksanvisning**  
**Käyttöopas**  
**Instrukcja obsługi**  
**Provozní návod**

**ESCHENBACH**

Deutsch .....	3
English.....	4
Français.....	5
Italiano.....	6
Español.....	8
Nederlands .....	9
Dansk .....	10
Svenska.....	11
Norsk.....	13
Suomi.....	14
Polski.....	15
Česki .....	16

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## English

### Safety instructions

- ▶ Risk of dazzling and injury! Do not look at the light source for long periods!
- ▶ Protect your desk lamp from impact or shock, moisture and excessive heat!  
Never place your desk lamp near a heat source or in direct sunlight.
- ▶ Do not use the desk lamp near containers filled with water!
- ▶ Risk of tripping! Make sure that the cable is not posing a danger!  
Do not wrap the cable around the table lamp!
- ▶ Ensure that other people, particularly children, are aware of these risks!

### Starting up

1. Set the desk lamp on a flat and stable surface.
2. Plug the mains power adapter into a properly wired power outlet.
3. Switch the lamp on or off with the pressure switch on the lamp base.
4. With the flexible elements at the top and bottom of the lamp arm, adjust the lamp head to your requirements.
5. To disconnect the unit from the mains power supply, unplug the mains power adapter from the wall outlet.

### Care instructions

#### Unplug the desk lamp from the power supply before cleaning!

Clean the device with a damp cloth. Do not use soap solutions containing plasticisers, alcoholic solvents or abrasive cleaners!

### Technical specifications

Stock number	16038
Input voltage of mains adapter	100 - 240 V ~
Output voltage of mains adapter	24 V =
Power	5 W
Luminous flux	380 lm
Colour temperature	5000 K
CRI	80
Energy efficiency class	G
LED lifespan	30000 hours

## Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made. Warranty claims are only accepted upon presentation of the sales receipt!

## Disposal

When the device reaches the end of its useful life, do not dispose of it with the normal household waste. This product is subject to the European directive in its currently valid version.

Dispose of the product or parts of the product through a certified disposal facility or your municipal disposal facility. Observe the currently applicable regulations. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

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Eschenbach Optik GmbH