easyPOCKET XL

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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English

Safety instructions

- **Danger of fire!**
Lenses used in optical equipment may cause considerable damage through their 'burning glass effect' if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!
- **Danger of blinding or injury** Never look at the sun through any optical equipment! Do not look directly at the light sources! Risk group 1 as per EN 62471. Retinal danger due to actinic light 400 nm to 780 nm.
- **Protect your magnifier from knocks, bangs, moisture and excessive heat.** Never place your magnifier on heaters or in the sunlight.
- **Ensure that other people, especially children, are aware of these dangers!**

- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Care Instructions

- When cleaning the easyPOCKET XL do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents! The lenses could be destroyed!
- Do not clean the easyPOCKET XL in an ultrasonic bath or under flowing water!
- Clean the easyPOCKET XL with a soft, lint-free cloth (e.g. with a cleaning cloth for glasses). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

Disposal

- This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.
- Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points.

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH