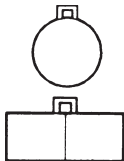
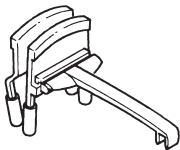
# laboCLIP

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**User Manual**

**Mode d'emploi**

**Istruzioni d'uso**

**Manual de instrucciones**

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**Bruksanvisning**

**Käyttöopas**

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## English

You have selected a quality product made by Eschenbach, which has been produced using the latest manufacturing processes in Germany; a top-class product "Made in Germany". We congratulate you on your choice.

All Eschenbach lenses are manufactured from PXM<sup>®</sup>, an optically pure plastic with user-friendly characteristics:

- ▶ Durable/unbreakable
- ▶ lighter than glass
- ▶ optically pure for best viewing quality

## Safety information

- **Risk of fire! Lenses used in optical equipment may cause considerable damage through their „burning glass effect“ if used or stored improperly. Ensure that optical lenses are never left uncovered in the sunlight.**
- **Danger of blinding or injury! Never look at the sun through any optical equipment.**
- **Protect your clip-on magnifier from knocks, strikes, moisture, and excessive heat.**
- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Reg-**

**ulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

## **Instructions**

laboCLIP is a very simple and efficient viewing aid for people wearing glasses and is mainly intended for farsighted glasses. For precise work or viewing the smallest details, laboCLIP is the ideal viewing aid. laboCLIP is a magnifying viewing aid that offers the greatest possible and therefore comfortable distance between the eye and the object.

laboCLIP comes in its own container with the lens component and four rubber guards.

To protect your glasses' lens, you need to fit the four guards [1] over the bridge [2] of the clip.

The clip [3] can then be attached to the frames of your glasses. (→ Fig. 2 and Fig. 3).

Depending on the required use, lenses with different magnifications and working distances (distance from lens to viewed object) can be clipped onto the holders [4], (→ Fig. 1)

You can order the following lenses individually:

<b>Magnification</b>	<b>Lens</b>	<b>Working distance</b>	<b>Order No.</b>
1.7 ×	binokular	about 400 mm	16455
2.0 ×	binokular	about 250 mm	16451

Magnification	Lens	Working distance	Order No.
2.5 ×	binokular	about 180 mm	16452
3.0 ×	binokular	about 160 mm	16453
4.0 ×	monokular	about 55 mm	16454
7.0 ×	monokular	about 32 mm	16457

## Care instructions

When the magnifying clip is not being used, it should be kept covered.

Clean the lens with a soft, lint-free cloth. For harder to remove stains (e.g. finger prints) use lukewarm water.

**Do not use any soap solutions containing softeners, alcohol-based solvents, or abrasive cleaning products.**

## Warranty

We are required by law to guarantee the functionality of the product described in this user manual with regard to defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or bumped, are not covered under the warranty. Warranty benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH