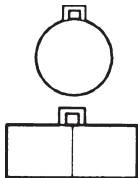
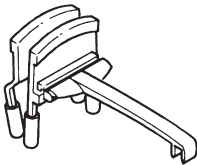
laboCLIP

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Operating instructions

Mode d'emploi

Istruzioni d'uso

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Brugsanvisning

Bruksanvisning

Käyttöohje

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English

You have selected a quality product made by Eschenbach, which has been produced using the latest manufacturing processes in Germany; a top-class "Made in Germany" product. We congratulate you on your choice.

All Eschenbach lenses are manufactured from PXM®, an optically pure plastic with user-friendly characteristics:

- durable
- lighter than glass
- optically pure for best viewing quality

Safety information

- **Risk of fire! Lenses used in optical equipment may cause considerable damage by their "burning glass effect" if used or stored improperly.**
Ensure that optical lenses are never left uncovered in the sunlight.
- **Danger of blinding or injury!**
Never look at the sun through any optical equipment.
- **Protect your glasses magnifier from knocks, strikes, moisture, and excessive heat.**

Instructions

laboCLIP is a very simple and efficient viewing aid for wearers of glasses and is mainly of use with farsighted glasses.

For precise work or viewing the smallest details, laboCLIP is the ideal viewing aid. laboCLIP is a magnifying viewing aid that offers

the greatest possible and therefore comfortable distance between the eye and object.

laboCLIP comes in its own container with the lens component and four rubber guards.

To protect your glasses' lens, you need to fit the four guards [1] over the bridge of the [2] clip.

The clip [3] can then be attached to the frames of your glasses. (→ Fig. 2 and Fig. 3)

Depending on the required use, lenses with different magnifications and working distances (distance from lens to viewed object) can be clipped onto the holders [4]. (→ Fig. 1)

You can order the following lenses individually:

Magnification	Lens	Working distance	Order No.
1.7 x	binocular	about 400 mm	16455
2.0 x	binocular	about 250 mm	16451
2.5 x	binocular	about 180 mm	16452
3.0 x	binocular	about 160 mm	16453
4.0 x	monocular	about 55 mm	16454
7.0 x	monocular	about 32 mm	16457

Care instructions

When the magnifying clip is not being used, it should be kept covered.

Clean the lens with a soft, lint-free cloth, for harder to remove stains (e.g. finger prints) use lukewarm water.

Do not use any soap solutions containing softeners, any alcohol-based solvents, or any abrasive cleaning products.

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Guarantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH