laboMED

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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 **ESCHENBACH**

Deutsch.....	3
English	6
Français.....	9
Italiano	13
Español	16
Nederlands.....	19
Dansk	23
Svenska.....	26
Norsk.....	29
Suomi	32
Polski	35
Česky.....	38
日本語	42

Deutsch

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English

You have purchased a quality product made with the latest production technology by ESCHENBACH in Germany; a brand product „made in Germany“. We would like to congratulate you on this decision.

Intended use

laboMED is used where precise work of detection of the smallest details is required. Depending on the field of application, lens parts of various magnifications can be fitted and different working distances (distance lens to object) can be applied.

Safety information


- ▶ **Danger of fire!**
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ **Danger of glare and injury!**
Never look directly into the sun or other bright light sources using optical devices!
- ▶ **Also inform other people and children especially of this safety measure!**

- ▶ **laboMED may not be used while driving automobiles.**
- ▶ **Protect your laboMED from impacts and excessive heat. Never lay your laboMED on a radiator or in direct sunlight.**
- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

The various lens parts are simply clipped onto the holder, as shown in illustration 1.

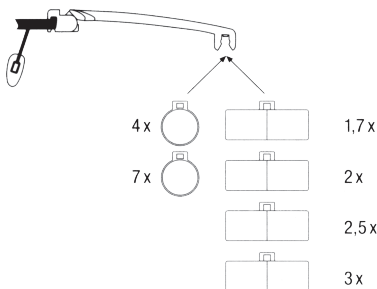Illustration 1

laboMED is supplied complete with **one** lens part in a storage box made of hard plastic. The box has several free compartments for additional lens parts. The following lenses can be ordered from your optician for your **laboMED**:

Order No.	Magnification	Lens	Working Distance
1645 1	2 x	Binokular	Approx. 250 mm
1645 2	2,5 x	Binokular	Approx. 180 mm
1645 3	3 x	Binokular	Approx. 160 mm
1645 4	4 x	Monokular	Approx. 55 mm
1645 5	1,7 x	Binokular	Approx. 400 mm
1645 7	7 x	Monokular	Approx. 32 mm


Fitting the lenses

To fit (rectangular) binocular lens parts pull the lens element carefully out of the holder and replace it by another one.

Because of their great magnification, **lens parts 4x and 7x** are used as **monocular elements only**, i. e. for a single eye.

For these lens parts the lens holder must be removed from the mounting, as indicated in illustration 2, and the lens part exchanged.

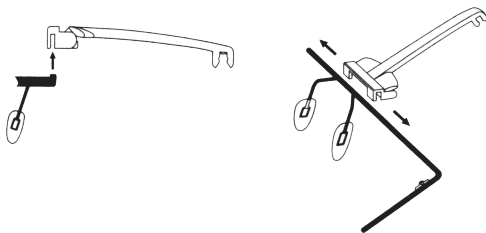Illustration 2

Instructions on Care and Maintenance

All ESCHENBACH lenses are made of PXM[®], an optically pure plastic material with a number of user-friendly properties:

- durable
- lighter than glass
- optically pure for the highest imaging quality

The PXM[®] lenses are coated in a special process are scratch-resistant, slightly demirrored and have a permanently antistatic coating. Apparent hairline cracks that may be detectable at the edge of the lens do not detract from the optical image. These are small line structures of the material that are production-related.

Should the **laboMED** lens part become soiled, please be sure to observe the following:

For cleaning purposes use a soft cotton or linen cloth. Always wipe the lens surface lightly. In case of bad soiling rinse the lens under hand-hot water. For cleaning purposes you should **on no account use alcoholic or organic solvents** as these can damage the lens.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH