maxDETAIL

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Bruksanvisning

Käyttöopas

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English

Your new acquisition is a quality product from the house of Eschenbach. This top-class 'Made in Germany' product was manufactured using the most up-to-date production methods available. Congratulations on your choice.

Intended use

The maxDETAIL allows you to read texts twice as large without straining your eyes. Crossword puzzles and working with your hands on hobby or work projects are made easy thanks to the two-fold magnification. The focus of its design was on user-friendliness and a modern style.

Safety instructions

- **Never look at the sun or other very bright sources of light through optical equipment!**

- **Risk of fire!**
Lenses used in optical equipment may cause considerable damage by their 'burning glass effect' if used or stored improperly. Make sure that optical lenses are never left uncovered in the sunlight.

- **Make sure that other people and especially children are aware of this.**
- **The maxDETAIL must not be used for driving in traffic!**
- **Protect your maxDETAIL from impact and excessive heat! Never place the maxDETAIL on heaters or in sunlight.**
- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Earpiece adjustment

The plastic earpieces ② can be bent in the desired position without warming. An internal metal wire ensures that the earpieces remain in this position.

Adjustment

The maxDETAIL is designed for a distance of about 40 cm and offers a dioptre adjustment (option for correcting defective Vision) of -3.00 dpt to +3.00 dpt. It can be adjusted for each eye individually. You can also use the dioptre adjustment to vary the glasses slightly to various distances.

Sit at the appropriate application distance and put on the maxDETAIL.

Keep your left eye closed and turn the knurled ring ① until your right eye can see in Sharp focus. Do the same with the other eye.

Notes

Under the following circumstances, it might not be possible to see the image perfectly:

- A very small or very large pupil spacing (eye spacing) can reduce the performance quality, because the image is not viewed through the optical zones of the maxDETAIL anymore.

- If you normally have glasses with dioptries of less than -3.00 dioptries or greater than +3.00 dioptries, the dioptre adjustment feature of the maxDETAIL cannot entirely compensate the defective Vision.
- If you usually wear glasses with cylindrical lenses (e. g. for astigmatism), it might not be possible to achieve perfect focus. This depends on the strength of the cylinder.

Care instructions

maxDETAIL is a high-quality, technologically advanced, complex optical aid and needs to be handled and cared for accordingly. For protection of the lenses against dust and external influences, we recommend that you store it in the case included.

- **Do not use alcohol or organic solvents to clean the maxDETAIL.**
- **Do not clean the maxDETAIL in an ultrasonic bath!**

Clean the maxDETAIL with a soft cotton or linen doth (such as deaning cloths for eye-glasses). Moisten the deaning doth to re-move more persistent stains (e.g. finger prints).

Technical data

| | |
|--|------------------------|
| Article number | 1624511 |
| Dioptre adjustment for each eye individually | -3,00 dpt to +3,00 dpt |
| Magnification | 2 × |
| Free working distance | 400 mm |
| Pupil spacing | 60 mm – 68 mm |
| Weigh | 49 g |

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product

described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH