maxDETAIL Clip

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Operating instructions

Mode d'emploi

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Bruksanvisning

Käyttöohje

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English

You have purchased a quality product from the Eschenbach company that was "Made in Germany" using state-of-the-art manufacturing processes. We congratulate you on your decision.

Intended use

maxDETAIL Clip is a magnifying attachment for clipping onto existing eyeglasses. This allows you to see twice as large without straining your eyes. Crossword puzzles and working within arm's reach, whether at home or at work, are easy thanks to the double magnification. Modern design and effective, easy use were the main priorities during development of this product.

Safety instructions

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices!
- ▶ Danger of fire!
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Also inform other people and children especially of this safety measure!
- ▶ maxDETAIL Clip may not be used while driving automobiles.
- ▶ Protect your maxDETAIL Clip from impacts and excessive heat. Never lay your maxDETAIL Clip on a radiator or in direct sunlight.

- ▶ Be particularly careful when using the product on eyeglasses with plastic lenses. When used improperly, the eyeglasses may be scratched since the lens of the product rests directly against the lens of the eyeglasses.
- ▶ The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

Before first use

Before first use, you have to place the enclosed rubber bar ends on the bars of the clip to prevent your glasses from being scratched.

Adjustment

The lens of the maxDETAIL Clip is designed for a working distance of about 400 mm from the eye.

To attach the maxDETAIL Clip to your eyeglasses, press the clip together.

When you wish to stop using the maxDETAIL Clip, simply flip the clip up, it is not necessary to remove it.

To allow an optimal line of sight on both large and small eyeglasses, the lens assembly on the clip can be moved vertically. To do this, it is necessary to unscrew the lens assembly with a small Philips screwdriver and rescrew it onto the clip arm at one of 5 possible positions.

The clip should generally lie in the lower line of sight through the eyeglasses so that you can still orient yourself in the room by looking through the upper part of the eyeglasses and over the clip.

If you cannot achieve good results with the Max-Detail Clip, this may be a result of one of the following situations:

- You may have selected a working distance that is too far or too short.
- You may be experiencing a vertical offset because you have attached the maxDETAIL Clip at too steep an angle.
- Your pupil distance may lie outside of the pupil distance range of the maxDETAIL Clip.

Care Instructions

Your maxDETAIL Clip is a technically and optically advanced vision product and requires appropriate handling and care. We recommend using the provided case for storage to protect the lenses from dust and external influences.

- **Do not use alcohol or organic solvents to clean the maxDETAIL Clip!**
- **Do not clean the maxDETAIL Clip in an ultrasound bath!**

Clean the maxDETAIL Clip with a soft cotton or linen cloth (e.g. lens cleaning cloth). In case of heavy soiling (e.g. fingerprints) moisten the cleaning cloth slightly.

Technical Data

- | | |
|-------------------------|---|
| ▶ Working distance: | 350 mm from lens crown
approx. 400 mm from eye |
| ▶ Pupil distance range: | 62 - 70 mm |
| ▶ Magnification: | 2 × |
| ▶ Field of vision: | approx. 85 mm |
| ▶ Weight: | 23 g |

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Eschenbach Optik GmbH