
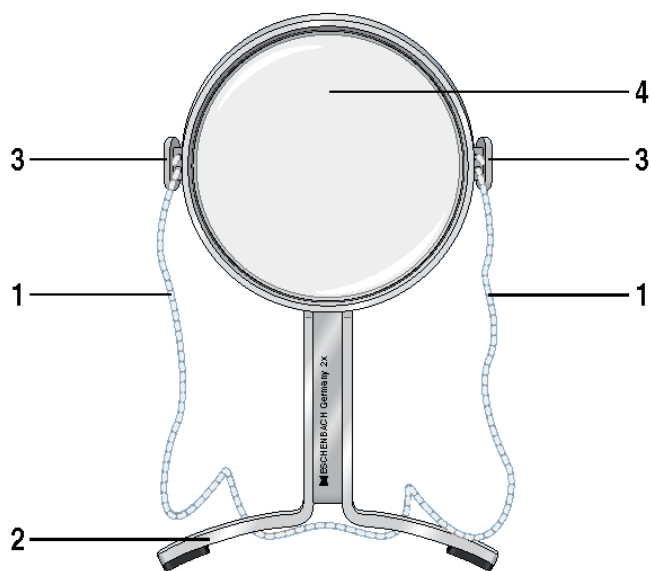
maxiPLUS ECONOMY

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
Руководство по эксплуатации
サービスマニュアル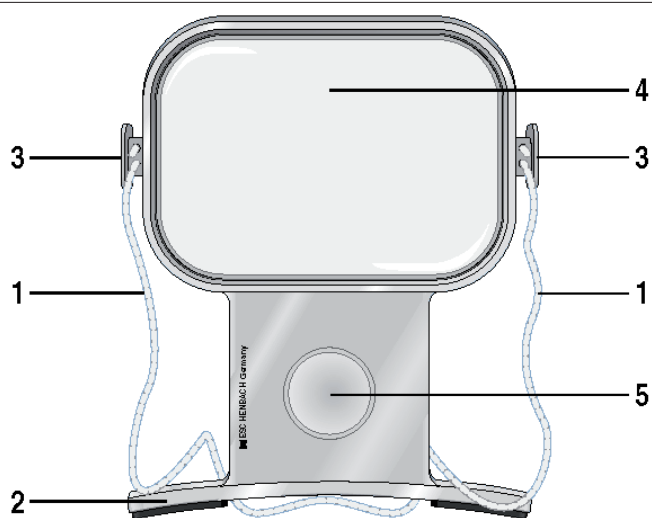

Deutsch.....	4
English	6
Français.....	9
Italiano	11
Español	14
Nederlands.....	16
Dansk	19
Svenska	21
Norsk.....	23
Suomi	26
Polski	28
Česky.....	31
Русский.....	33
日本語	35

Deutsch

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English

Dear Customer:

Congratulations on purchasing this Eschenbach hands-free magnifying glass, a quality product "Made in Germany". We hope you enjoy using it every day.

Intended use

This quality Eschenbach hands-free magnifying glass is a visual aid for everyday situations, such as crocheting, knitting, sewing, embroidering, threading needles, as well as model building and many other hobbies – even nail care.

Whatever you are doing, it leaves both hands free to work with.

As the market leader for magnifying glasses and reading glasses, we at Eschenbach can look back on many years of experience, which is reflected in the quality, handling, and design of our products.

Safety instructions

- **Risk of fire!**

Lenses used in optical equipment may cause considerable damage through their "burning glass effect" if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

- **Danger of blinding or injury!**

Never look at the sun through any optical equipment.

- **Protect your hands-free magnifying glass from knocks and impacts, moisture, and excessive heat. Never leave your magnifying glass on a radiator or in the sun.**

- **Make sure others, especially children, are aware of this danger!**

- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Care tips

Keep the magnifying glass covered when you are not using it. Clean the lenses with a soft, lint-free cloth, such as a cloth for cleaning glasses. If they are very dirty (for example with fingerprints), clean the lenses with lukewarm water.

- **Do not use any soapy solutions that contain softeners, alcohol-based solvents, or abrasive cleansers. This may damage the lenses beyond repair.**

Instructions for use

- ▶ Hang the cord [1] of the hands-free magnifying glass around your neck.
- ▶ Place the curved support [2] of the magnifying glass on your chest or stomach so that the magnifying glass is at a right angle to your body.
- ▶ By winding the cord [1] around the cord holders [3] on the sides of the magnifying glass, you can adjust the working distance as needed.

The maxiPLUS 2678 hands-free magnifying glass has two lenses.

The large lens [4] (140×100 mm), which magnifies to twice the normal size, is for various craft and hobby tasks, and the small lens [5] ($\varnothing 35$ mm) is suitable for tasks requiring higher precision, such as threading a needle.

The ECONOMY 2677 hands-free magnifying glass has one lens.

Technical specifications

Item	Lens size	dpt	Magnification
maxiPLUS	100×140 mm	2.6	$2 \times$
	$\varnothing 35$ mm	11.2	$4 \times$
ECONOMY	$\varnothing 100$ mm	3.2	$2 \times$

Do you need more magnifying power? Many more useful Eschenbach visual aids are available from your optician.

Warranty

We are required by law to guarantee the functionality of the product described in this user manual with regard to defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or bumped, are not covered under the warranty.

Warranty benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH