menas LUX

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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English

You have purchased a quality product made with the latest production technology by ESCHENBACH in Germany; a brand product „Made in Germany“. We would like to congratulate you on this decision.

Safety information

- ▶ **This product is not a toy! Do not let children handle it without supervision!**
- ▶ **Danger of fire!**
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!

- ▶ **Risk of blinding and injury!**
Never look directly into the sun or other bright light sources using optical devices! Do not look directly at the light source!
- ▶ **Protect your menasLUX from impacts and excessive heat. Never lay your menasLUX on a radiator or in direct sunlight.**
- ▶ **Check the batteries regularly. Leaking batteries can cause damage to the device and it could lead to irritation if the leakage makes contact with the skin.**
- ▶ **Remove the batteries from the battery compartment as soon as they are spent or if the device is to be unused for a long period of time.**
- ▶ **Also inform other people and children especially of this safety measure!**

- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Using the menasLUX

menasLUX is handy magnifier placed directly on the text to be read so that is presented larger.

1. Place the menasLUX on the text that is to be magnified.
2. Press the button ❶, to switch the light on.
3. Move the menasLUX over the text to read it.
4. After use, switch the light off by pressing the ❶ button once again.

Changing batteries

menasLUX is powered by a 1.5 V battery (Size AA, Mignon) for the LED light. Should the brightness of the light diminish, it means the battery is discharged and must be replaced. Proceed as follows.

1. Switch the light off.
2. Open the battery compartment **3**, by removing the battery cover on the underside of housing in the direction of the arrow.
3. Replace the battery **2** with a new one of the same type. Pay attention to the correct polarity.
4. Close the battery compartment by sliding the battery cover in the opposite direction of the arrow on the housing.

Technical Data

- System Magnification: 3.0x, 12 diopters
- Battery: 1.5V-Battery, Size AA, Mignon

Disposal

This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points.

Instructions on Care and Maintenance

Clean the menasLUX with a soft, lint-free cloth (i.e. eye glasses polishing cloth). With heavy soiling (i.e. fingerprints) clean the menasLUX carefully with a moist cloth. Never use alcoholic and/or organic solvents to clean the menasLUX as these can destroy it.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing or material defects. In the event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH