miniframe BIF0

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Bruksanvisning

Käyttöopas

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 **ESCHENBACH**

Deutsch.....	3
English	4
Français.....	5
Italiano	6
Español	8
Nederlands.....	9
Dansk.....	10
Svenska.....	11
Norsk.....	13
Suomi	14
Polski	15
Česky.....	16
日本語.....	17

Deutsch

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English

You have purchased a quality product "made in Germany" from the Eschenbach company. We congratulate you on your decision.

Important safety instructions, please read carefully!

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices!
- ▶ Danger of fire! Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Also inform other people and children especially of this safety measure!
- ▶ miniframe bifo may not be used while driving automobiles.
- ▶ Only intended for close-up vision and reading.
- ▶ Regular professional eye examinations are essential for optimal vision correction and evaluation of the health of your eyes.
- ▶ Not suitable for seeing at a distance.
- ▶ Do not use as an eye protection device.
- ▶ The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

Intended use

These bifocal prismatic magnifying glasses are intended for close use. The base glass has a refractive index of +3.0 diopters and is designed for normal reading needs. Depending on the model, the segment has a refractive diopter value of +4.5, +6.0 or +7.5, for special reading needs such as small print or detailed work.

Care Instructions

Do not use alcohol or organic solvents to clean the “miniframe bifo”! Clean the “miniframe bifo” with a soft cotton or linen cloth (e.g. lens cleaning cloth). In case of heavy soiling (e.g. fingerprints) moisten the cleaning cloth slightly.

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH