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**mobilent / mobilent LED**

**User Manual**

**Mode d'emploi**

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**Bruksanvisning**

**Käyttöohje**

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## English

### Safety instructions

- ▶ This product is not a toy!  
Do not let children handle it without supervision!
- ▶ Risk of asphyxiation!  
This product contains small parts that could be swallowed!
- ▶ Risk of fire!  
Lenses in optical devices can cause significant damage by the focusing of light to generate heat if handled and stored improperly! Ensure that you never leave uncovered optical lenses lying in direct sunlight!
- ▶ Risk of glare and injury!  
NEVER look directly into the sun or other bright light sources with optical devices!

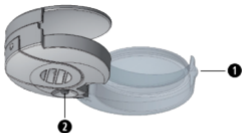
- ▶ Protect your magnifier against impacts of all kinds, moisture and excess heat. Never place your magnifier on radiators or in direct sunlight!
- ▶ Ensure that other people, particularly children, are aware of these risks!

The following also applies for the mobile LED:

- ▶ Never look directly at the light source on the underside of the housing!  
Risk group 1 according to EN 62471.
- ▶ Check the condition of the battery regularly.  
Leaking batteries can cause damage to the device and could lead to irritation if the leakage makes contact with the skin.
- ▶ Remove the battery from the battery compartment as soon as it is spent or if the device is not to be used for a long period of time.
- ▶ Never expose the magnifier to moisture.
- ▶ The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

## Use

- ▶ Hold the magnifying glass housing with one hand while you swing out the magnifying glass with the other hand using the gripping aid ❶.

- ▶ When using the mobile LED, you can compensate for insufficient illumination of the reading material by turning on the LED lighting via the button ❷ on the underside of the housing.
- ▶ After use, push the magnifying glass back into the magnifier housing to protect it from dust and scratches. If necessary, turn the lighting off again via the button ❷.

## Care instructions

- ▶ Do not use any soap solutions that contain softening agents, any alcoholic or organic solvents or any abrasive detergents to clean the mobilent / mobilent LED! The lenses could be destroyed!
- ▶ Do not clean the mobilent / mobilent LED in an ultrasonic bath or under running water!
- ▶ Clean the mobilent / mobilent LED with a soft, lint-free cloth (e.g. with a cleaning cloth for glasses).  
For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

## Disposal (mobilet LED only)

- ▶ This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. 
- ▶ Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points. 

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Eschenbach Optik GmbH