Bedienungsanleitung
User Manual
Instructions d'utilisation
Manuale d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Bruksanvisning
Käyttöohje
Instrukcja obsługi
Návod k obsluze
取扱説明書

DE

EN

FR

IT

ES

NL

DA

SV

NO


FI

PL

CS

JA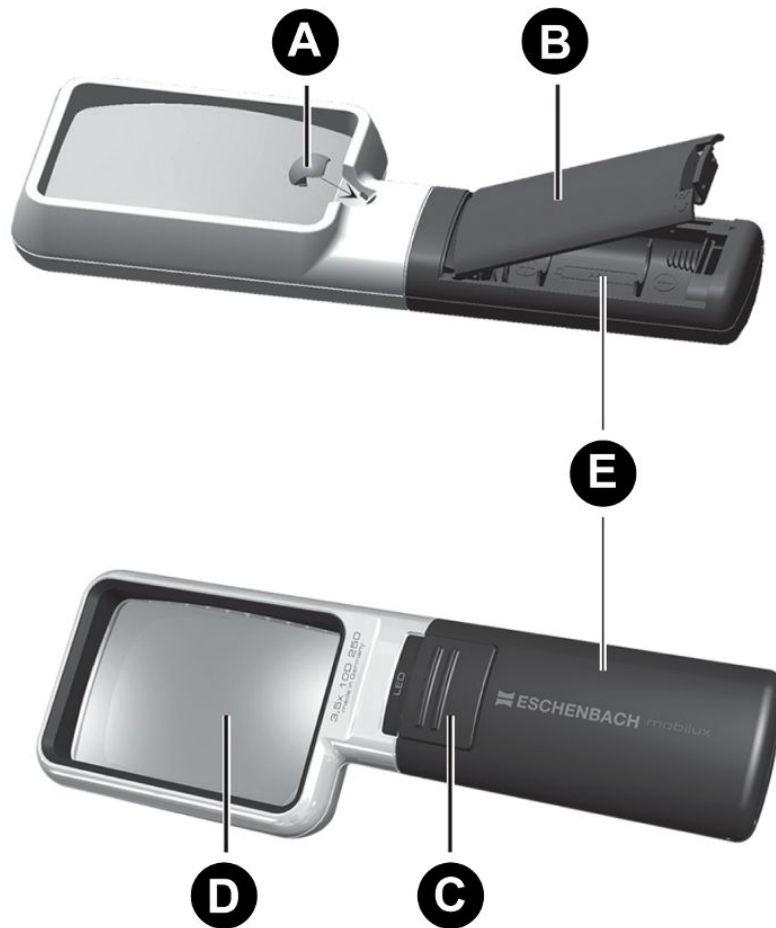

Leuchtlupe mobilux® LED

| Nr. No. | ∅ | ⊕ | ♀ 👁 |
|---------|------------|--------------|--------|
| 15122 | 90 × 54 mm | 8 dpt/3× | 250 mm |
| 15112 | 60 mm | 12 dpt/3× | 250 mm |
| 15113 | 75 × 50 mm | 10 dpt/3,5× | 250 mm |
| 15114 | 75 × 50 mm | 16 dpt/4× | 180 mm |
| 151141 | 60 mm | 16 dpt/4× | 150 mm |
| 15115 | 58 mm | 20 dpt/5× | 140 mm |
| 15116 | 58 mm | 24 dpt/6× | 100 mm |
| 15117 | 35 mm | 28 dpt/7× | 60 mm |
| 151110 | 35 mm | 38 dpt/10× | 40 mm |
| 151112 | 35 mm | 50 dpt/12,5× | 40 mm |

Deutsch

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About this manual

This manual applies to the products listed on page 3.

Read through this manual carefully before using the product described in it. Keep the manual in a safe place. If you pass on the product, give the recipient this manual too.

The following signs and symbols are used in this manual and/or on the product:

| Symbol | Meaning |
|---|--|
|  | General warning of a hazard |
|  | Dispose of the product according to the local guidelines |
|  | Manufacturer: see back page |

Safety instructions

 The product is designed and built to state-of-the-art standards and complies with the relevant European directives. Nonetheless, residual hazards exist. To avoid hazards, you must follow the safety instructions. Make sure that other people are also aware of these hazards.

Children can injure themselves when handling the product. Never allow unsupervised children to play with packaging material and the product.

The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the production

outside the EU member states, the respective country-specific directives must be observed.

Risk when handling the product

- Never leave optical lenses lying in the sun without a cover. The burning glass effect of the lenses poses a fire hazard.
- Never look at the sun through optical products.

Never look directly into the product's light sources (risk group 1 to EN 62471).

Risk of property damage

- Protect the product from impact or shock, moisture and excessive heat.
- Never place the product near heat sources or in direct sunlight.
- Do not clean the product under flowing water or in an ultrasonic bath.
- Do not use soap solutions containing plasticisers, alcohol-based solvents or abrasive cleaning products to clean the product.

Risk when handling batteries

- Check the batteries regularly. Leaking batteries can cause damage to the product and it could lead to irritation if the leakage makes contact with the skin.
- Remove the batteries from the battery compartment as soon as they are spent or if the product is to be unused for a long period of time.
- Make sure that the batteries are disposed of properly. Do not dispose of batteries in the household waste.

Intended use

The product is intended for magnifying a reading and working area. It is only suitable for indoor use in dry rooms.

Only use the product as described in this manual. Any other use is considered improper and can lead to property damage or personal injury. Do not make any changes to the construction of the product. No liability is assumed for damage caused by improper or incorrect use.

Product description

Illuminated magnifier

| | |
|-----------------------------|------------------------------------|
| A LED filter cap | B Battery compartment cover |
| C [On/off] switch | D Lens |
| E Eyelet for lanyard | |

Operation

Inserting / changing the batteries

- Always use 2 new batteries of the same type.
- Note the position of the respective positive terminal.

1. Open the battery compartment cover **B**.
 2. Remove the spent batteries from the battery compartment.
 3. Insert the new batteries into the battery compartment.
- ⇒ The negative terminal of the battery is pointed in the direction of the battery compartment's spring.

4. Close the battery compartment cover.

Switching the illuminated magnifier on and off

1. Push the [On/Off] switch **C** in the direction of the lens **D**.
⇒ The LED light is switched on.
2. Push the [On/Off] switch away from the lens.
⇒ The LED light is switched off.

Using the illuminated magnifier

If you normally wear glasses for reading, you do not need to remove them. The illuminated magnifier is designed to also be used together with your reading glasses.

1. Switch on the light.
2. Hold the illuminated magnifier with the lens centrally above the text to be magnified.
The optimal imaging quality is reached if you maintain the distance between the magnifier and your eyes given for the product. The distances in mm can be found on page 2 or above the switch on the product.
3. Switch off the light after use.

Adjusting the colour temperature and light intensity

- ✓ The light is switched off.
- Insert one of the 3 LED filter caps  supplied into the recess in front of the light outlet.
- ⇒ The orange and the yellow LED filter caps change the colour temperature.
- ⇒ The transparent filter concentrates the light and increases the light intensity.

Technical data

| Light source | |
|------------------------|--------------------------------|
| Lamp | LED (risk group 1 to EN 62471) |
| Power supply | |
| Battery | 2 × Mignon (AA/LR6) / 1.5 V |
| Ambient conditions | |
| Operating temperatures | 0 °C ... 40 °C |

Cleaning and care

1. Make sure that the illuminated magnifier is switched off.
2. Clean the lens with a soft, lint-free cloth, for example, a microfibre cloth.
3. For heavier soiling, e. g. finger-marks, moisten the cloth slightly. Do not use soap solutions containing plasticisers, alcohol-based solvents or abrasive cleaning products.
4. When not in use, keep the illuminated magnifier in the case supplied.

Disposal

Disposal of electrical equipment

This product is subject to the currently valid version of the European Directive on Waste Electrical and Electronic Equipment. Do not dispose of the product in the normal household waste bin.

Dispose of the product or parts of it through a certified disposal facility or your municipal disposal facility. Note and comply with the currently applicable regulations. If in doubt, contact your disposal management facility. Dispose of all packaging materials in an environmentally friendly manner.

Disposal of batteries and rechargeable batteries

Do not dispose of batteries and rechargeable batteries in the household waste bin.

To dispose of the device, remove the batteries or rechargeable batteries from the device and hand them in to an official collection point.

Declaration of Conformity

The product meets all requirements of the relevant EU directives and the national regulations. We confirm this with the CE marking. You can retrieve the declaration of conformity for this product on the following website: <http://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>.

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Trade Name:mobilux® LED
Model No.: 15112, 15113, 15114, 151141, 15115, 15116, 15117, 151110, 151112, 15122

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.
22 Shelter Rock Lane
Danbury, CT 06810
United States
Tel.: (800) 487-5389
Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interfer-

ence to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

Guarantee

Within the parameters of the applicable statutory provisions, Eschenbach Optik GmbH provides a guarantee for the function of the product described in this manual with regard to any faults arising due to manufacturing defects or material defects. In the event of damage due to improper handling, including damage from a fall or impact, no guarantee claims can be made. The guarantee can only be claimed by presenting proof of purchase.

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