
mobilux[®] LED

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
Руководство по эксплуатации
サービスマニュアル

ESCHENBACH

Deutsch.....	4
English.....	9
Français.....	13
Italiano	18
Español	23
Nederlands.....	28
Dansk.....	33
Svenska.....	38
Norsk.....	42
Suomi	47
Polski	51
Česky.....	56
Русский.....	60
日本語	66

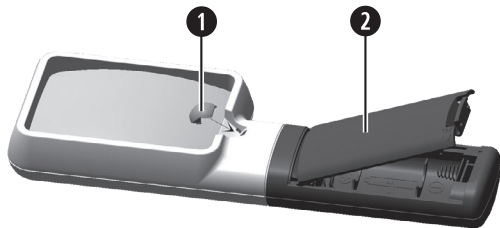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English

Introduction

You have acquired a brand name product from Eschenbach that has been manufactured in Germany as per the most modern production processes. Before the first use you should have read through these instructions attentively so as to become familiar with all of the device's functions. mobiluxLED is intended for use as a visual aid, in that the product presents you with a magnified image.

Intended use

This product is designed exclusively for providing enlarged images of text, pictures, or other object details.

Safety Instructions

- ▶ **This product is not a toy! Do not let children handle it without supervision!**
- ▶ **Risk of asphyxiation! This product contains small parts that could be swallowed!**
- ▶ **Danger of fire! Lenses in optical devices can cause significant damage by the focusing of light to generate heat if handled and stored improperly! Take care that you never leave uncovered optical lenses lying in direct sunlight!**
- ▶ **Danger of glare and injury! NEVER look directly into the sun or other bright light sources with optical devices! Never look directly at the light source on the underside of the housing! Risk group 1 as per EN 62471.**

- ▶ **Protect your illuminated magnifier against impacts of all kinds, moisture and excess heat. Never lay your illuminated magnifier on a heating radiator or in direct sunlight!**
- ▶ **Check the batteries regularly. Leaking batteries can cause damage to the device and it could lead to irritation if the leakage makes contact with the skin.**
- ▶ **Remove the batteries from the battery compartment as soon as they are spent or if the device is to be unused for a long period of time.**
- ▶ **Never subject the mobiluxLED to wetness!**
- ▶ **Ensure that other people, particularly children, are aware of these risks!**
- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Initial device operation

For equable illumination of your reading material this mobiluxLED is equipped with a high-performance LED, which is powered by 2 batteries (AA/LR6). Before the first use, you must insert the batteries.

1. To do this, open the battery compartment cover (②).
2. Insert 2 batteries into the battery compartment as per the alignments shown therein. Pay attention to observance of the correct polarities, otherwise the device will not work.
3. Close the battery compartment.

Using the mobiluxLED

You can use mobiluxLED with your reading glasses.

1. Hold the mobiluxLED with the lens centrally over the text that is to be magnified.
2. The optimal imaging quality with mobiluxLED is to be obtained when you maintain the distance to the lens determined for you for the magnification used by you. This value (in mm) is to be found in the chapter 'Technische Daten / Technical Data / Caractéristiques techniques' on page 3,  ←  (Spacing lens-eye) as well as above the switch on your mobiluxLED.
3. Switch the LED light on by sliding the shifter (❸) in the direction of the lens (❹). If you require stronger illumination, fit the transparent LED filter cap (❶) of the 3 enclosed LED filter caps, as shown in the left figure on page 3.
4. After use, switch the LED light off by sliding the shifter (❸) in the direction away from the lens (❹).

Note: If the LED flickers, the batteries are drained and must be exchanged for new ones. In this case, proceed as detailed in the chapter 'Initial device operation'.

If the colour of the light is unpleasant for you, you can place one of the enclosed LED filter caps (❶, orange or yellow) over the LED, as shown in the left figure on page 3.

To protect the lens your mobiluxLED is supplied with a case.

To enable you to hang your mobiluxLED around your neck it has an eyelet located on the end of the handle and through which a lanyard, which is optionally available, can be threaded. (Stock.-No. 15981).

Care Instructions

- ▶ When cleaning the mobiluxLED do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents! The lenses could be destroyed!
- ▶ Do not clean the mobiluxLED in an ultrasonic bath or under flowing water!
- ▶ Clean the mobiluxLED with a soft, lint-free cloth (e.g. with a cleaning cloth for glasses). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

Disposal

- ▶ This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.
- ▶ Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In the event of damage due to improper handling, including damage from falls or impacts, no warranty claims can be accepted. Warranty claims can only be accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH