Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Návod k obsluze
サービスマニュアル

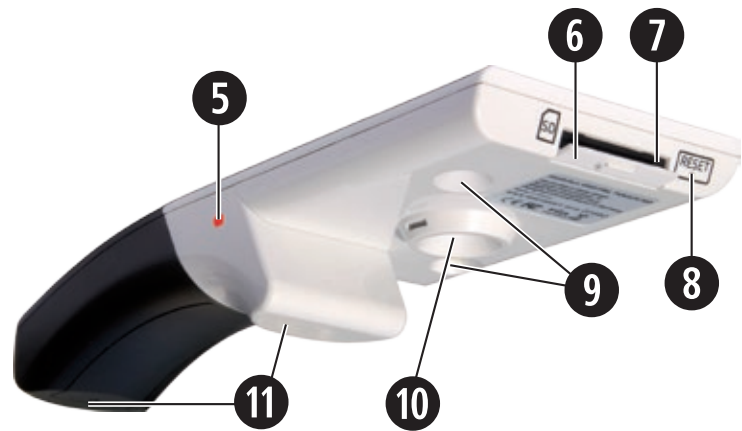
CE

mobilux® **DIGITAL** Touch HD
16511

ESCHENBACH

Deutsch.....	4
English.....	25
Français	45
Italiano	66
Español.....	87
Nederlands	108
Dansk.....	129
Svenska	149
Norsk.....	170
Suomi.....	190
Polski.....	210
Česky.....	230
日本語.....	250

Deutsch

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English

Dear customer,

we congratulate you on the purchase of this electronic magnifying glass for hand-held or contact use - a quality product made by Eschenbach - and we hope you enjoy using the device on a daily basis.

As the market leader for magnifying glasses and reading glasses, we at Eschenbach are able to look back on many years of experience which are mirrored in the quality, handling and design of our products.

Firmware version

This manual applies to firmware version "MD2_fw_v2.0.0" and higher.

Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images.

Before initial use

Before using the device for the first time, please read the entire manual. This is the only way to get to know all the functions on the device.

Safety instructions

- ▶ Risk of glare and injury: Never look directly at the light source on the underside of the housing.
- ▶ Do not allow children to handle electronic devices without supervision.
- ▶ Only charge the reader with the mains adapter supplied.
- ▶ Risk of suffocation and accidental swallowing: Keep packaging, films and packing parts away from children.
- ▶ Check the device and the mains adapter with power cable for visible damage. If there is any damage disconnect the connected power unit from the power supply immediately. Ask a qualified specialist to check the product before you use it again.
- ▶ Lay the power cord so that no one can trip over it.
- ▶ Never open the device housing. There are no serviceable parts whatsoever inside. Opening the device will void the guarantee.
- ▶ The installed lithium-ion batteries may only be removed or replaced by authorised specialist personnel.

- ▶ Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.
- ▶ Never expose the device to moisture.
- ▶ The light source contained in this luminaire shall only be replaced by the manufacturer or his service agent or a similar qualified person.
- ▶ Risk of pinching: Be careful not to pinch your fingers when inserting the stand.
- ▶ Protect the device against impact or knocks and excessive heat. Never place the device on radiators or in direct sunlight.
- ▶ Do not store the device in places exposed to excessive heat, such as in a parked car.
- ▶ For domestic use only.
- ▶ This device may not be operated in a medical environment together with other devices that are used for life support.
- ▶ Ensure that other people, particularly children, are aware of these risks.
- ▶ The device is a Class I medical device and complies with Directive 93/42 EEC, Annex IX, Regulation 12. When using the device outside the EU member states, the respective country-specific directives must be observed.

Package contents

- ▶ Electronic reader mobilux DIGITAL Touch HD (in the remaining text also referred to simply as "reader")
- ▶ Charger (with plugs for EU, GB, US, AUS)
- ▶ Stand
- ▶ USB cable
- ▶ SD card (supplied preinstalled)
- ▶ Case, Cleaning cloth, Lanyard
- ▶ Operating instructions

Operating elements

1. Touch-sensitive display
2. On/Off button
3. USB/charging socket
4. Eyelet for lanyard
5. LED charging indicator
6. Cover for SD card slot
7. SD card slot
8. Reset opening
9. LED lights
10. Camera
11. Contact surface

Operation

Charging the internal battery:

before you use the mobilux DIGITAL Touch HD for the first time, you must fully charge the internal battery.

To charge, connect the USB plug of the charger with the USB/charger socket (❸) of the reader. Then connect the charger with the plug appropriate for the operating location and plug the charger into a power outlet. Charging takes about 2 hours. The charging indicator LED (❹) will light up red during the charging process. Once the charging process is complete, the LED charging control lamp goes out. Disconnect the charger from the electrical outlet and the reader once charging is completed.

You can use the reader during the charging process. Using the reader while charging will, however, prolong the charging time.

Switching on/off

Press and hold the On/Off button (❷) for a second to turn on mobilux DIGITAL Touch HD and to switch to live image mode. The enlarged image of the reading material will appear on the display (❶).

On the upper edge of the display, icons appear for 5 seconds to display the battery charge level, the last set zoom level and the display brightness.

Battery fully charged

75% charge

50% charge

25% remaining charge, the icon lights up red for 5 seconds on the upper edge of the display

Remaining run time approx. 3 minutes, the icon lights up red permanently on the upper edge of the display

To switch off the device, press and hold the On/Off button for one second. The current zoom, display mode and brightness settings will be saved.

To prolong the battery charge of the internal battery, the reader switches off automatically if it is not moved for more than 3 minutes. If you want to use the reader again after this time, switch it on again.

Reading with the device

Switch on the device and place its contact surface (⑪) on the reading material and move it in the direction of reading while reading.

The reader is provided with a high depth of field. In order to enable smaller magnifications as a 4-fold magnification, and therefore larger visible areas, you can also hold your mobilux DIGITAL Touch HD in your hand a short distance away from the reading material, or you can insert it in the stand supplied. Insertion in the stand is described on the next page („Writing with the appliance“).

Writing with the device

Use the supplied stand if you wish to write under the magnifier. Insert your mobilux DIGITAL Touch HD into the stand as shown here until it audibly engages:

Separate your mobilux DIGITAL Touch HD from the stand in the same order.

Save display image as file

By briefly tapping on the display, three icons will appear on the right-hand edge of the display:

icon for the "Save display image as file" (screenshot)

icon for "Set zoom"

icon for "Display mode"

Tap briefly on the  icon to save the current live image as a file on the SD card inserted in the appliance. The image is then shown on the screen. The screenshots are numbered sequentially on the SD card. Tap again on the  icon to return to live image mode.

Images can only be shown on the display as still pictures if no SD card has been inserted.

Touch the  icon for 2 seconds to display to change to the "Display image files" mode.

If you tap on the  icon, you will move back one menu level. If you tap on the  icon, the menu will close and the reader returns to live image mode.

Adjusting the Magnification

Tap repeatedly on the  icon to scroll through the the seven available zoom levels 4x, 5x, 6x, 8x, 10x, 12x and 15x. These factors apply to contact use. In hand-held use, the zoom levels are lower than stated. The currently set zoom level is shown on the upper edge of the display for approx. 2 seconds.

Setting the display mode

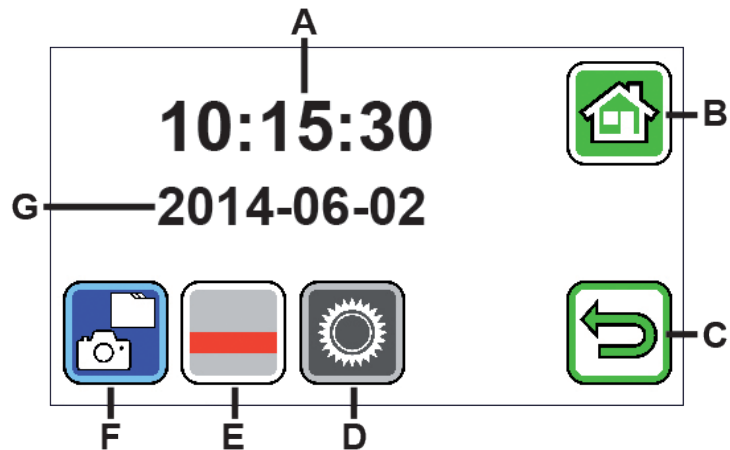
The mobilux DIGITAL Touch HD is equipped with a range of display modes so that you can achieve the best possible image. Tap repeatedly on the  icon to scroll through the various display modes from true-colour mode via high-contrast black/white and white/black displays to false-colour black/yellow and yellow/black displays.

Touch the  icon for 2 seconds to switch on/off the LEDs (9) that illuminate the reading material on the underside of the device.

Tap again briefly on the display to hide the three icons.

Opening the menu

Press the display gently for 3 seconds to call up the following menu:

A - Time display

B - "Close menu" button

C - "One menu level back" button

D - "Call up settings" button

E - "Activate/deactivate guide lines" button

F - "Display image files"

G - Date display

Display image files

If you tap this button briefly, the two most recently recorded image files are displayed as thumbnails. Tap one of the preview images to display it at full screen size. Tap the display to delete the image file, to zoom in or to change the contrast. Next to the B and C icons, the icons ,  and  are displayed one below the other. Tap again on the display to restore the icons.

Delete file

To delete a file, tap on this icon and answer the subsequent security prompt by tapping on the  icon. The file will then be permanently deleted from the SD card. If you do not want to delete the file, tap on the  icon to cancel the operation.

Tap repeatedly on the  icon to scroll through the the seven available zoom levels 4×, 5×, 6×, 8×, 10×, 12× and 15×.

Tap repeatedly on the  icon to scroll through the already described display modes.

Tap on the  icon to move back one menu level. You will then see the thumbnails as well as the icons described below.

Tap on the  icon to go back 10 files in the preview.

Tap on the  icon to display the thumbnails of the previous two files.

Tap on the  icon to display the thumbnails of the following two files.

Tap on the  icon to move forward 10 files in the preview.

Tap on the  icon to close the menu and return to the live image mode.

Show reading line

When you tap this button, the live image appears and the function icon appears in the centre of the image:

 (reading line activated),  (reading row activated) or  (reading line or reading row deactivated). Tap the icon for the required function, which is then immediately shown on the display. If you tap on the  icon, you will move back one menu level.

Call up device settings

If you tap this button briefly, you can then adjust the device settings described below. Here, too, you can use the  icon to return to the previous menu level. Use the  to close the menu and return to the live image display.

Setting the signal tone

Depending on the desired tone, tap the  or  icon. Tap on  if you prefer to have no acoustic confirmation of your input.

Setting display brightness

Set the desired display brightness here: 50%, 75% or 100%.

Setting continuous zoom

If you have selected this function, two icons will appear in the middle of the display next to each other: On the left, the  icon (for continuous magnification) and on the right the  icon (for incremental zoom). The currently active setting has a yellow background, the inactive setting has a grey background. Tap the associated icon to activate the required function.

 icon: In Live Image Mode, changes to the zoom take place variably if you tap the  or the  icons.

 icon: Tap the  icon to change the zoom (only zooming in) in Live Image Mode. If the photo function is switched off, use the  and  icons to zoom in and out.

You can use the  icon to return to the previous menu level. Use the  icon to close the menu and return to the live image display.

Switching the photo icon on and off

If you do not require the photo function, you can remove the photo icon from the selection menu on the right-hand display border. To remove, tap the  icon. To add, tap the  icon. If the photo icon is switched off, the selection menu in Live Image Mode changes to  (zoom in),  (zoom out) and  (contrast).

Switching between right-hand/left-hand use

You can use this function to switch the appliance over between left-hand and right-hand use. Tap this icon, and then tap the  icon to set the appliance up for left-handed users. The display is then turned 180° so that it can be read by left-handed users. Tap the  icon to set up the appliance for right-handed users.

Setting the date and time

After calling up this function, you can tap on the parameters for the date and time to increase or decrease the parameters (now displayed white) using the /  buttons. Tap on "24H" to switch between 24-hour and 12-hour mode.

Setting the line guide function

After calling up this function, you can set the the reading line or reading bar (shadow bars that darken the upper and lower part of the display black). Tap on the  icon to switch between reading line and reading bar. If a red reading line is displayed in front of the current live image, you can move this as required using the  /  buttons. If shadow bars are displayed at the top and the bottom of the display instead of the reading line, you can adjust their size as required using the  /  buttons.

Update firmware

If you call up this function, the firmware version currently installed on the appliance is displayed on the left as the "Current Version", and the firmware version to be installed is shown on the right as the "Update Version". To update the reading appliance firmware, tap the  icon, to cancel the process, tap the  icon.

To update the software, the software version to be installed must have been previously copied onto the SD card.

Using the mobilux DIGITAL Touch HD on the computer

You can transfer the live image from the reader onto your computer screen. To do this, you must first install the "mobilux Digital" software supplied onto the computer. The installation guide can be found on the pre-installed SD card. The following operating systems are required for running mobilux Digital: Windows Vista, Windows 7, Windows 8 or Windows 10. mobilux Digital may also run on other Windows platforms, but this cannot be guaranteed. We can accept no liability for the data integrity of computers running operating systems other than those mentioned here.

3 USB functions

If you connect the reading device to a computer USB connection and switch it on, you will see the following 3 symbols on the reading device display:

Tap the  icon to set up the contents of the SD card in the reader as a new removable drive. Then proceed with the required file operation.

You will find a PDF installation guide called "Software installation manual" in the "manuals" folder. Double-click the file name to open the manual using the default program for viewing PDF files.

Follow the installation instructions corresponding to the operating system installed on your computer.

Tap the  symbol to enable communication between the reading device and the mobilux Digital software previously installed and started. The reading appliance live image is shown in the program window. The following four icons are shown on the top left in the window:    . The functions for the first three icons that can be called up with the mouse are equivalent to the functions already described here. If you tap the  icon, the version number of the installed software will be displayed. If you tap the unfolding icon to the right of the  icon and then select "Manual", the operating instructions for the mobilux DIGITAL Touch HD will be opened from the SD card as a PDF version in the linked standard program.

Tap the  symbol in order to work with the reading device while the internal battery is being charged using the USB cable.

To change the USB mode, tap the display briefly to show the symbols if necessary. Then tap bottom left on the symbol for the current USB mode in order to re-display the selection of 3 symbols as described above. Then tap the symbol for the required USB mode.

Reset

If the reader stops working as described in this manual, it may be due a fault that you can rectify yourself by resetting the reader. Insert a small, pointed object carefully into the opening labelled RESET (8) and press the switch contained within.

Care Instructions

- ▶ When cleaning the mobilux DIGITAL Touch HD do not use any soap solutions that contain softening agents, alcoholic or organic solvents, and do not use abrasive detergents!
- ▶ Do not clean the mobilux DIGITAL Touch HD in an ultrasonic bath or under flowing water!
- ▶ Only clean the mobilux DIGITAL Touch HD with a soft, dry cloth. For heavier soiling (e.g. fingerprints), moisten the cloth slightly.
- ▶ If you are not using the mobilux DIGITAL Touch HD, always keep it in the hard foam case supplied.

Technical data

Display size:	11.0 cm (4.3")
Zoom range:	Contact operation: 4×, 5×, 6×, 8×, 10×, 12×, 15× Hand-held at approx. 4 cm: 1.6×, 1.9×, 2.2×, 3.0×, 3.7×, 4.5×, 5.6× with stand: 2.5×, 2.9×, 3.5×, 4.7×, 5.8×, 7.2×, 8.8×
Operating / charging time:	Approx. 3 / 2 hours
System voltage of the charger:	100 V-240 V~, 50-60 Hz
Temperature range:	10°C-40°C
Weight:	230 g

This manual is also available on the Internet at:

<https://www.eschenbach-sehhilfen.com/de-DE/278/produktuebersicht/detail/150/mobilux-reg-DIGI-TALTouch-HD>

Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made. Warranty claims are only accepted upon presentation of the sales receipt!

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Eschenbach Optik GmbH