noves MONO

Bedienungsanleitung
Operating instructions
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Návod k obsluze
Руководство по эксплуатации
サービスマニュアル

 **ESCHENBACH**

Deutsch.....	3
English.....	5
Français	7
Italiano	9
Español.....	11
Nederlands	13
Dansk	15
Svenska	17
Norsk	19
Suomi.....	21
Polski	23
Česki.....	25
Русский.....	27
日本語	29

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High-magnification readying glasses from Eschenbach.

We congratulate you on your purchase of novel MONO reading glasses from Eschenbach.

The novel MONO reading glasses are supplied in a monocle design. They come with both an optical lens and an opaque lens.

The lenses are protected by a cera-tec® hard coating. Scratch-resistance is a decisive criterion for the quality and long-life of the lens.

Safety information

► Risk of fire!

Lenses used in optical equipment may cause considerable damage through their "burning glass effect" if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

► Make sure that other people and especially children are aware of this!

► Danger of blinding or injury!

Never look at the sun through any optical equipment!

► Protect your reading glasses from impact and excessive heat! Never leave your reading glasses on heaters or in the sun!

► These reading glasses are only appropriate for reading and must not be used for driving in traffic!

► The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

Use

1. Use a suitable source of lighting which provides even and non-reflective illumination for your reading material.
 2. Put on your reading glasses.
 3. To find the correct reading distance, hold the reading text close in front of the reading glasses.
 4. Slowly move the reading material away from your reading glasses until the text has a sharp resolution. This is the correct distance for reading.
- The text will not be clear if the reading distance is too close or too far away from you.

Reading distance for various reading glasses

- The reading distance is dependent on the magnification of the reading glasses being used.

Diopters / magnification	Reading distance
12 Diopters / 3 ×	approx. 83 mm
16 Diopters / 4 ×	approx. 65 mm
20 Diopters / 5 ×	approx. 50 mm
24 Diopters / 6 ×	approx. 42 mm

If you have any questions, please contact your specialized dealer.

Care instructions

- ▶ **Do not use alcohol or organic solvents to clean the reading glasses!**
- ▶ **Do not clean the reading glasses under running water or in an ultrasonic bath!**

Clean the glasses with a soft cotton cloth- or linen cloth (e. g. a glasses cleaning cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. finger marks).

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH