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**DE**

**Operating Instructions**

**EN**

**Mode d'emploi**

**FR**

**Istruzioni d'uso**

**IT**

**Manual de instrucciones**

**ES**

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**NL**

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**DA**

**Bruksanvisning**

**SV**

**Bruksanvisning**

**NO**

**Käyttöohje**

**FI**

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**PL**

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**CS**

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**JP**

**Deutsch**

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**English**

About these instructions .....	19
Safety.....	19
Technical specifications .....	21
Product description .....	22
Operating and functional elements.....	23
Download the app .....	23
Switching on and off.....	23
Commissioning .....	24
optaro app.....	25
Troubleshooting .....	30
Care instructions .....	31
Disposal .....	32
Applicable standards.....	32
Warranty .....	34
Trademarks.....	34

## About these instructions

These instructions contain important information for the safe use and proper cleaning of your product.

Read these instructions and the safety instructions (you will find these in a separate booklet in the packaging) carefully and thoroughly before using this product.

Keep these instructions for future reference.

These instructions are based on the app version "1.0.0".

## Safety

### Intended use

This electronic reader is intended exclusively for enlarging and reading or viewing texts or images. To ensure proper function, the reader must be connected to a compatible iPhone via a separately available smartphone protective cover or a universal stand.

This product is intended for use in a domestic environment (including care homes, workplaces, schools and other educational establishments).

#### Important information about the usage environment

The device connects to the iPhone via WiFi. The use of WiFi and the corresponding radio connections are defined by national regulations.

This device is approved for use in the following countries: DE, AT, BE, BG, CY, CZ, DK, EE, EL, ES, FI, FR, HR, HU, IE, IS, IT, LI, LT, LU, LV, MT, NL, NO, PL, PT, RO, SE, SI, SK, CH, USA, UK(NI)

In Japan, use is only permitted indoors. Please observe the respective national regulations.

No liability is accepted for improper or unauthorised use in non-approved countries.

## Safety instructions

### Important information for your safety

Read and follow the following safety instructions carefully!

Make other people - especially children - aware of these dangers.

- Risk of glare and injury! Do not look into the light sources on the underside of the housing.
- Do not allow children to handle electrical appliances unsupervised. The product is not a toy!
- Choking hazard and danger of swallowing: Keep packaging, foils and packaging parts away from children.
- Protect the device from impact or shock. Handle the device with care.
- Lay the USB cable so that nobody can trip over it.

- Check the device and the cable for visible damage. In the event of damage, disconnect the connected mains adapter from the power supply immediately. Have the product checked by qualified specialists before using it again.
- Risk of trapping: Be careful not to trap your fingers when folding in the stand.
- Never open the device housing. Opening it will invalidate the warranty.
- Do not use the device while walking.
- We shall not be liable for the correctness of the contents of the "Tell me" function. This is particularly important when processing sensitive information such as the dosage of medication.

### **Correct handling of rechargeable batteries**

- The built-in battery may only be removed or replaced by authorised specialist personnel.
- Caution: Risk of explosion if the wrong battery is used.
- Dispose of used batteries in accordance with the legal requirements.
- Never throw a rechargeable battery into a fire.
- Never crush or cut a rechargeable battery.
- Never leave a battery in an environment with extremely high temperatures, as this can lead to an explosion or the release of flammable liquids or gases.
- Never expose a battery to extremely low air pressure, as this can lead to an explosion or the release of flammable liquids or gases.

### **Notes on the usage environment**

- Do not charge the device in places with excessive heat. High ambient temperatures can affect the charging process.
- Never expose the device to moisture or wetness. The device is not waterproof!
- Do not store the device in places with excessive heat, such as in a parked car.
- Protect the device from excessive heat and naked flames. Never place the device on radiators or in direct sunlight.
- In a medical environment, this device must not be operated with other life support equipment.

### **Scope of delivery**

- optaro
- USB cable (Type-C to Type-C)
- Operating instructions

## Technical specifications

<b>Magnification range</b>	3.3x or 3.7x to 15x (depending on the end device used)
<b>Display mode</b>	True colour 14 contrast-enhanced contrast modes
<b>Camera</b>	Fixed focus at a distance of 24mm
<b>Resolution</b>	1920 x 1080
<b>Illumination</b>	LED
<b>Recommended power source</b>	USB-PD (5V=1,8 A/9 V=1,0 A/12 V=0,75 A)
<b>USB connection</b>	Type-C
<b>WiFi SSID</b>	See type plate and operating instructions
<b>Conducted power</b>	< 9 dBm
<b>Radio network</b>	WiFi 5745 - 5825 MHz (EU, UK, US), 5180-5240 MHz (Japan)
<b>Operating time</b>	Approx. 2 hours
<b>Charging time</b>	Approx. 1.5 hours
<b>Dimensions (folded)</b>	103 x 72 x 20 mm
<b>Weight</b>	120 g
<b>Operating temperature range</b>	0°C - 35°C

## Product description

The optaro electronic reader is a medical device for people with impaired vision and is used to magnify reading material (e.g. text and images).

EN

*optaro with smartphone protective cover or an universal stand*

The product is connected to a compatible iPhone using a separately available protective smartphone case.

The connection to the smartphone is established via a local WiFi connection. No internet connection is required for this.

The reader is powered by its own battery. The battery is charged via USB-C and has an operating time of approx. 2 hours.

The foldable stand holds the reader in position.

The product is controlled via the “optaro app”.

The reading material is recorded via the reader’s camera. The app magnifies the recorded data on the smartphone display. The reader enables 3.3x to 15x magnification (depending on the mobile device), various false colour modes and is equipped with a fixed-focus camera.

Other functions include the photo function, a read-aloud function of individual words or groups of words, the display of a reading aid and an overview display.

## Compatible smartphones

optaro protective cases are available for a variety of different iPhone models. A list of all compatible devices can be found under the following link:

[www.eschenbach-vision.com/en-GB/products/product/optaror-16533](http://www.eschenbach-vision.com/en-GB/products/product/optaror-16533)

## Operating and functional elements

EN

- A Swivelling camera
- B RESET button
- C Status LED
- D On / Off button
- E LED illumination
- F Retractable stand
- G Eyelet for neck strap
- H Charging indicator LED
- I USB-C charging port

### Download the app

The optaro app is freely available in the Apple App Store and can be downloaded there.

The app can also be downloaded using the QR code on the back.

#### **Important information about the QR code**

- Keep the QR code in a safe place for future use (e.g. changing the device)!

### Switching on and off

To switch on, press and hold the on/off button (D) for one second. To switch optaro off, press and hold the on/off button (D) for two seconds.

#### **Automatic switch-off**

The camera module switches off automatically after 3 minutes if the 'optaro App' is not connected.

# Commissioning

## Information on the protective covers/universal stand

The reader must be used with the appropriate protective cover or the universal stand.

EN

### To set up the optaro reader, proceed as follows:<sup>1</sup>

1. Download the optaro app from the Apple App Store or via the QR code on the back.
2. Switch on the reader by pressing the on/off button (D).
  - The status LED above lights up blue when the camera module is ready.
3. Start the optaro app on your smartphone.
4. Press “Next” in the optaro app.
5. Confirm that you have read and agree to the terms of use by ticking the box. You must agree to the terms of use in order to use the app and the reader.
6. Press “Next”.

### Note

To be able to use the optaro with the app, a local WiFi connection is established between the iPhone and optaro. iOS requires permission to find WiFi devices in the vicinity.

For the initial setup, you will be asked to enter the optaro SSID once. This can be done in two ways:

- Manual entry of the SSID (1)
  - Scanning the QR code on the type plate (2)
    - ① If you want to establish the connection by scanning the QR code, you must authorise the optaro app to access your iPhone's camera.
7. Enter the SSID in the field provided above or use the scan function (2).
  8. Press “Next”.
  9. The connection must be authorised two times during the setup process.
    - The connection is established with the camera module.
    - After a successful connection, the status LED (C) of the camera module changes from blue to green.
    - The main screen of the app is displayed.

---

<sup>1</sup> For information on installing the optaro and protective cover, please refer to the installation instructions for the corresponding protective cover.

- Now place the smartphone in the protective cover.
- ✓ The reader is ready for use.

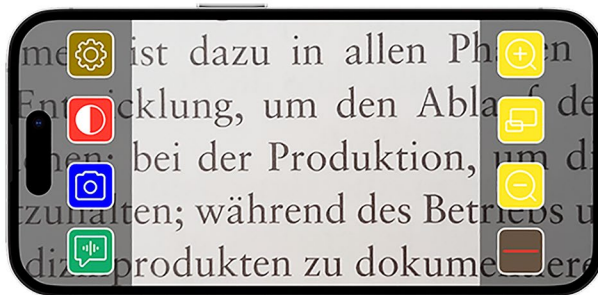
### Establishing the connection between the smartphone and optaro

Each time the app is started, the connection between the optaro camera module and the iPhone must be authorised.

## optaro app

### Display screen

The main screen of the optaro app is structured as follows and offers various functions:

Icon	Name	Meaning
	Zoom in	Increasing the zoom factor
	Overview function	Switches between the smallest and the previously set magnification
	Zoom out	Reducing the zoom factor
	Activate/deactivate reading support	Activates/deactivates reading support
	“Tell me” function	Starts the read aloud function for the central reading area
	Take a photo	Take a photo and switch to photo mode
	Activate/deactivate false colours	Switch between the selected false colour
	Settings	Opens the “Settings” menu

## Functions

### False colours

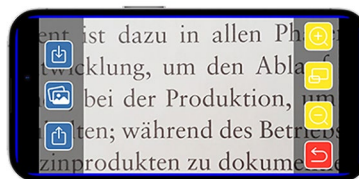
To display the camera image in false colours, switch between the various false-colour modes or return to the true colour display, press repeatedly

### Take a photo

In photo mode (marked by the blue frame at the edge of the screen), you can save the current screen section to your gallery, open your gallery or share the displayed screen section.

optaro saves the photos in a separate album named “optaro” (iOS requires permission to access the photo library).

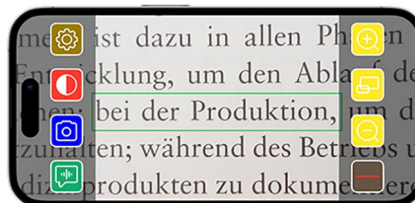
	Save	Saves the displayed screen section as a photo in the gallery.
	Photo gallery	Opens the “Photo gallery” of the iPhone
	Share photos	Opens the menu for sharing and sending the photo
	Back button	Exits the photo mode and switches back to the live mage

### “Tell me” function

You can use the “Say a word” function to have a central group of words read aloud. The central word group is read aloud by pres-

sing the  button. The area read aloud is highlighted with a green border.

### Important information on voice output

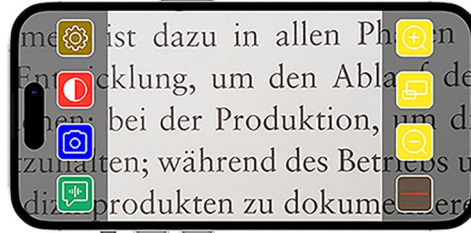
The function is not available in all languages. <sup>1</sup>

<sup>1</sup> Available languages are: German, English, French, Italian and Spanish

### Zoom in / Zoom out

You can use the “Zoom in / Zoom out” function to enlarge or reduce the current screen section or switch between the smallest and previously set magnification.

Zoom in      Enlarges the screen section

Overview function      Switches between the smallest and the previously set magnification

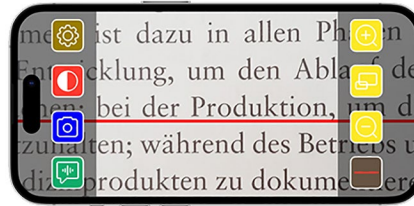
Zoom out      Reduces the screen section

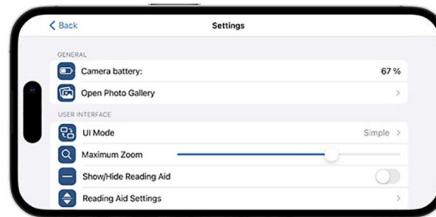
### Show / hide reading support

You can use the “Show and hide reading support” function to show reading support in the current screen section. You can activate and deactivate the reading support by pressing the  button.

### Settings

Press  to call up the settings menu. The following settings can be made:

### Camera battery

You can view the current battery status here.

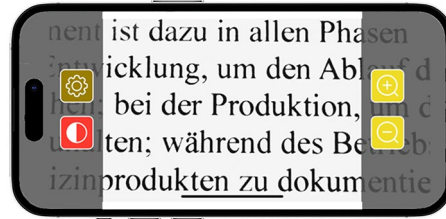
### Open photo gallery

You can access your albums in the photo gallery. The ‘optaro’ album shows all photos taken with optaro. The photos can be enlarged or reduced accordingly. You also have the option of sending the photos (see chapter "Take a photo" on page 26).

### Operating mode

Here you have the option of changing the operating mode. The 'Advanced' mode is displayed by default. The 'Simple' operating mode can be set by pressing the icon. Only four icons are displayed here (settings, contrast, zoom in and zoom out).

### Max. magnification

Here you can set the maximum magnification using the slider. A maximum magnification of 15x is displayed by default.

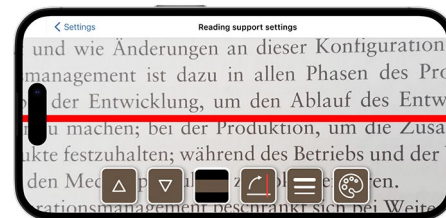
### Show/hide reading support

Reading support (reading line / reading bar) can be activated or deactivated here. In advanced operating mode, the reading support can also be displayed or hidden via .

### Reading support settings

After clicking on the icon, the following overview appears.

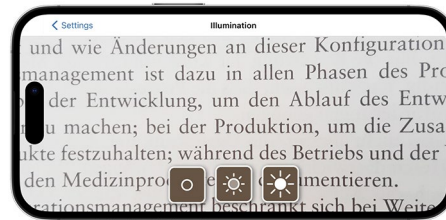
	Move reading line upwards	You can use this button to move the reading line upwards. Alternatively, you can also move the reading line by sliding it with your finger.
	Move reading line downwards	You can use this button to move the reading line downwards. Alternatively, you can also move the reading line by sliding it with your finger.
	Switch between reading line and reading bar	In addition to the reading line, a reading bar can also be set. This reading line can be adjusted in height by pressing the  and  .
	Vertical/horizontal reading support	Switching from horizontal reading line / reading bar to vertical reading line / reading bar can be set here.

	Line thickness	The thickness of the reading line can be adjusted by pressing this button. Three different thicknesses can be selected.
	Colour of the reading line	The colour of the reading line can be adjusted by pressing this button. In addition to the red reading line (set as standard), a black or grey reading line can also be used.

## Illumination

You can use this button to set the LED illumination. The medium illumination is preset by default. If the illumination level is increased, this will result in a shorter battery life.

	Illumination 50%	Setting the illumination to 50%
	Illumination 100%	Setting the illumination to 100%
	Illumination off	Setting the illumination to 0%

## False colours

To select false colour combinations, press this button. The default setting is black/white and white/black. The set false colour combinations are displayed with a tick. A total of 14 different false colour combinations can be set.

## Update firmware

Here you can check whether the latest firmware is available. If a new firmware version is available, a message to carry out the update will appear here.

## Reset to factory settings

Press this button to reset the app to the factory settings. If the camera module is connected to the app (status LED lights up green), reset the camera module at the same time (see chapter "Reset camera module" on page 31).

## Operating instructions

The operating instructions can also be viewed on the iPhone. To do this, press this button.

## About

Displays further information about the manufacturer and the app.

## Privacy policy

The privacy policy can also be viewed on the iPhone. To do this, press this button.

## Terms of use

The terms of use can also be viewed on the iPhone. To do this, press this button.

## Licence information

The licence information can also be viewed on the iPhone. To do this, press this button.

## Troubleshooting

If the device no longer functions as described in these instructions, there may be a fault that you can rectify yourself.

There are five possible solutions that you can try:

1. Repeat the procedure that did not work again.
2. Press the RESET button B under the folding foot of the camera module. Then close and restart the app (as described here: [www.support.apple.com/de-de/109359](http://www.support.apple.com/de-de/109359)). Repeat the procedure that did not work again.
3. Reset the camera module and the app to the factory settings (see chapter "Reset camera module" on page 31). Then repeat the initial setup:
4. Delete and reinstall the app. Reset the camera module to the factory settings (see chapter "Reset camera module" on page 31). Then repeat the initial setup:
5. In exceptional cases, you may have to switch off your iPhone and reinstall the app.

If the above assistance does not solve your problem, please contact your specialist dealer for further support.

### Function of the RESET button

Using the RESET button B only leads to a hard reset of the camera module. No settings are reset.

## Reset camera module

EN

To reset the app and the camera module, proceed as follows and in this order:

1. Open the settings menu  in the app.
2. Reset the app using the “Reset to factory settings” function.
  - This function also resets the camera module if it is connected correctly.
3. If the camera module is not connected, the camera module must also be reset.
4. To do this, press and hold the On / Off button D until the status LED C flashes.
  - ✓ The components are reset.

### Important information on resetting

During the initial setup, the factory default passwords are automatically overwritten to ensure the highest possible data security. From this point onwards, your camera module can only communicate with your app.

If one of the two components is reset, it will use the factory default password again. Communication with the other components is possible again after both components have been reset. Always reset both components together.

## Care instructions

Clean the device with a soft, lint-free cloth. If the optical elements (camera, LEDs) are heavily soiled, e.g. with fingerprints or similar, lightly dampen the cloth.

- Avoid cleaning the device with:
  - Soap solutions that contain plasticisers
  - Alcoholic or organic solvents
  - Abrasive cleaning agents
  - Ultrasonic baths
  - Running water
- Failure to observe the instructions may result in damage to the device.

## Disposal

This product is subject to the European WEEE directive on old electrical and electronic equipment in its currently valid version.

EN

Do not dispose of your old device with household waste, but via an authorised waste disposal company or your municipal waste disposal facility.

Observe the regulations applicable in your country. If in doubt, contact your waste disposal organisation. Dispose of all packaging materials in an environmentally friendly manner.

## Applicable standards

The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation 2017/745, Annex VIII, Chapter III, Rule 13. If the device is used outside the EU member states, the respective country-specific directives must be observed.

Hereby, Eschenbach declares that the radio equipment type (Art.-No. 16533) is in compliance with Directive 2014/53/EU.

The applicable declaration of conformity can be found on the following website:

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SL18AA Burnham, Bucks

Internet: [www.associatedoptical.com](http://www.associatedoptical.com)

Hereby, Eschenbach Optik GmbH declares that the radio equipment type optaro camera is in compliance with Radio Equipment Regulations 2017. The full text of the UK declaration of conformity is available at the following internet address:

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## Supplier's Declaration of Conformity

### 47 CFR § 2.1077 Compliance Information

#### Unique Identifier

Product Name: optaro

Model No.: 16533

#### Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

EN

#### FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**MODIFICATION:** Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

The portable device is designed to meet the requirements for exposure to radio waves established by the Federal Communications Commission (USA). These requirements set a SAR limit of 4 W/kg averaged per ten gram of tissue. The highest SAR value reported under this standard during product certification for use when properly worn on the limbs.

EN

## **Warranty**

We guarantee the function of the product described in these instructions within the framework of the statutory provisions with regard to any occurring defects that are attributable to manufacturing faults or material defects. Damage caused by improper handling, including damage caused by dropping or impact, is not covered by the warranty. Warranty only by proof of purchase.

## **Trademarks**

USB® is a registered trademark of USB Implementers Forum, Inc.

iPhone and App Store are trademarks of Apple Inc. registered in the USA and other countries and regions.

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- Tras conectarse correctamente, el LED de estado © del módulo de cámara cambia de azul a verde.