powerlux

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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English

You have purchased a quality product from the Eschenbach company that was “Made in Germany” using state-of-the-art manufacturing processes. We congratulate you on your decision.

The lens of high-quality, unbreakable PXM® material is protected with a cera-tec® hard coating. Scratch-resistance is an important aspect of the quality and useful life of the lens.

Safety Instructions

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices! Never look directly into the light source.
- ▶ Danger of fire! Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Protect your magnifying light from impacts, moisture and excess heat. Never lay your magnifying light on a radiator or in direct sunlight.
- ▶ Inform other persons, particularly children, of these dangers.
- ▶ The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.

The device components

- ① Lens
- ② Lighting switch
- ③ Battery compartment
- ④ Battery compartment cover

Setting Up

1. Remove the device from the packaging.
2. Open the battery compartment ❸ on the rear panel of the device. For this press on the area with the two small bars and slide the battery compartment cover ❹ away from the device.
3. According to the positioning diagram in the battery compartment ❸ place 3 × 1.5 V Batteries, Type Mignon, size AA, into the compartment ❸.
4. Close the battery compartment ❸.

Using powerlux

1. move powerlux with the lens ❶ over the text that is to be magnified.
2. Powerlux has two SMD high performance LEDs for even illuminating of the reading area. To switch on the built-in LED lighting please press the button ❷. After approx. 30 minutes the lighting switches off automatically. If, after this time, you want to continue using the lighting press the button ❷ again to switch the lighting back on.
3. After approx. 30 minutes the lighting switches off automatically. As a result of this, unnecessary depletion of the batteries will be avoided if you forget to switch powerlux off after use. If you use powerlux for less than 30 minutes you can switch the lighting off by pressing the button ❷.

Care Instructions

Clean the housing with a moist cloth. Clean the lens ❶ with a soft, lint-free cloth (e.g. glasses cleaning cloth). In case of heavy soiling (e.g. fingerprints), clean the lens ❶ carefully with a damp cloth.

Do not use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! These could destroy the lens ❶.

Technical Data

► Optics:

Refractive power	14 diopters	20 diopters	28 diopters
Magnification	3.5 ×	5 ×	7 ×
Diameter	58 mm	58 mm	58 mm

► Stock Number:

Light colour: Cold white (approx. 8000 K)	158614	158620	158628
Light colour: Warm white (approx. 3000 K)	1586141	1586201	1586281

► Illumination value: 4000 lx

► LED - Class 1 EN 60825-1 in its currently applicable version.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

Disposal

This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility,  not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points.

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Eschenbach Optik GmbH