Bedienungsanleitung
User Manual
Instructions d'utilisation
Manuale d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Bruksanvisning
Käyttöohje
Instrukcja obsługi
Návod k obsluze
取扱説明書

DE

EN

FR

IT

ES

NL

DA

SV

NO

FI

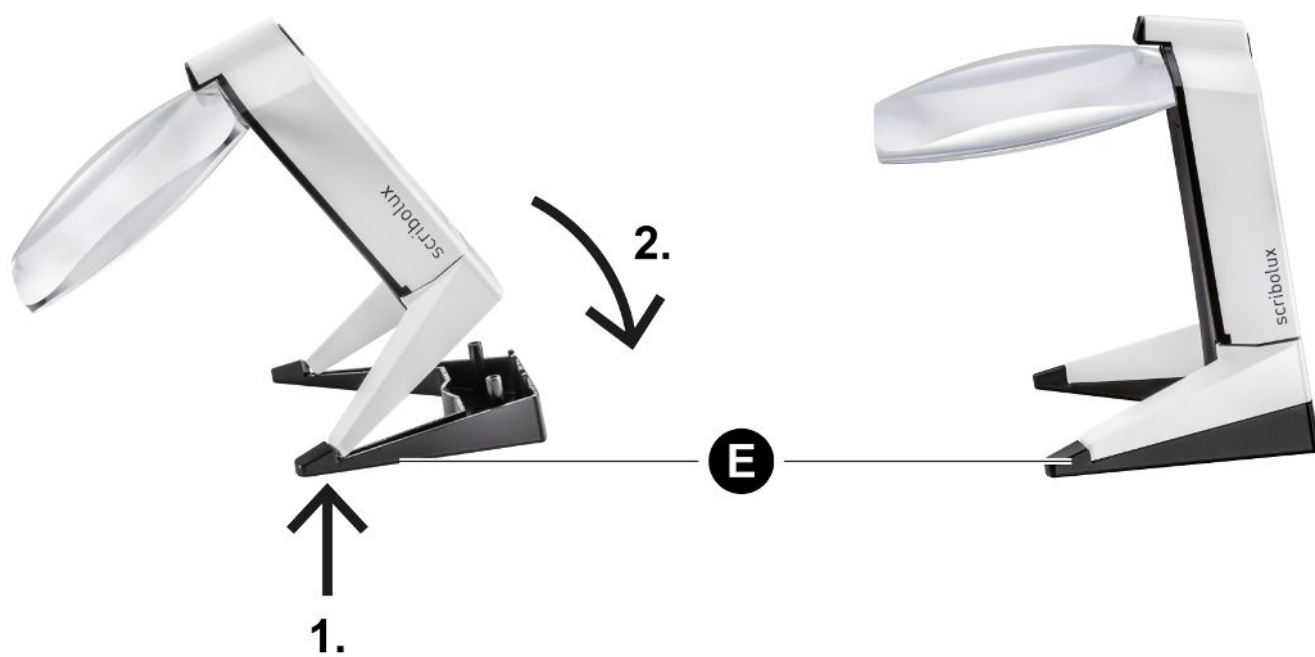
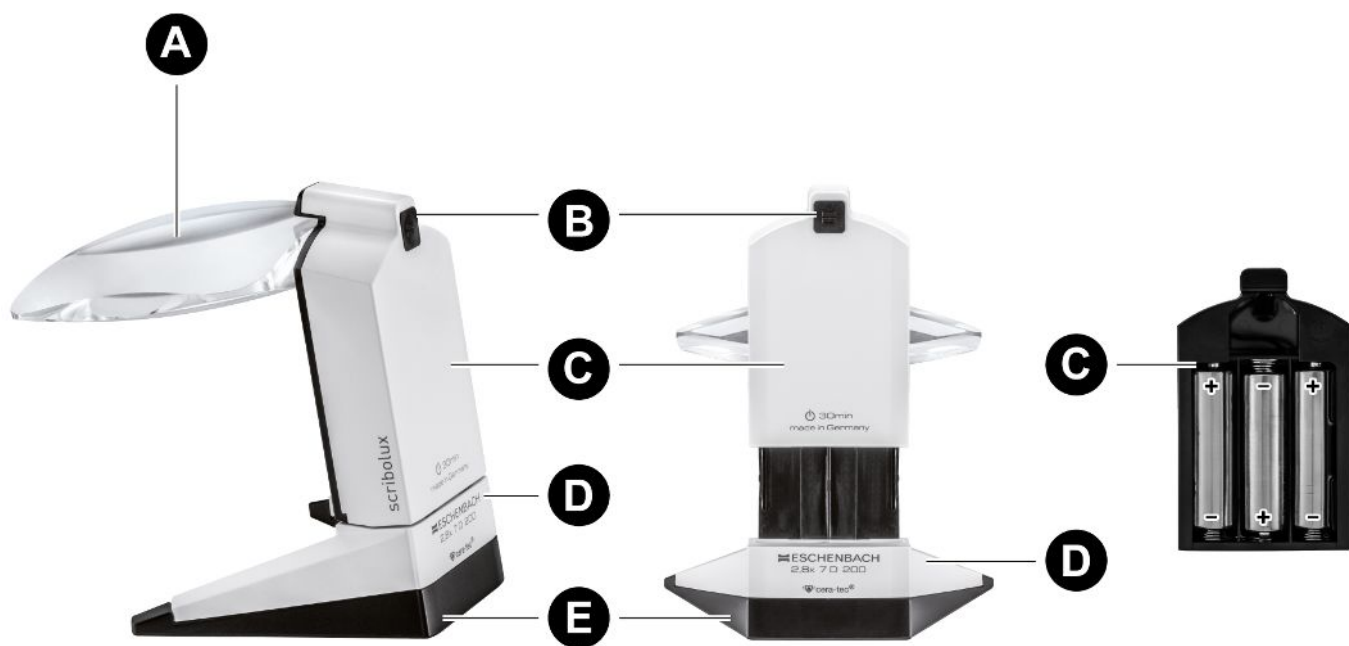
PL

CS

JA

scribolux

Deutsch

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English

About this manual

This manual applies to the following product:

Illuminated magnifier scribolux

Nr.
No.

Magnification 2.8 ×

156514

Read through this manual carefully before using the product described in it. Keep the manual in a safe place. If you pass on the product, give the recipient this manual too.

The following signs and symbols are used in this manual and/or on the product:

Symbol	Meaning
	General warning of a hazard
	Dispose of the product according to the local guidelines
	Manufacturer: see back page

Safety instructions

 The product is designed and built to state-of-the-art standards and complies with the relevant European directives. Nonetheless, residual hazards exist. To avoid hazards, you must follow the safety instructions. Make sure that other people are also aware of these hazards.

Children can injure themselves when handling the product. Never allow unsupervised children to play with packaging material and the product.

The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the production outside the EU member states, the respective country-specific directives must be observed.

Risk when handling the product

- Never leave optical lenses lying in the sun without a cover. The burning glass effect of the lenses poses a fire hazard.
- Never look at the sun through optical products.

Never look directly into the product's light sources (risk group 1 to EN 62471).

Risk of property damage

- Protect the product from impact or shock, moisture and excessive heat.
- Never place the product near heat sources or in direct sunlight.
- Do not clean the product under flowing water or in an ultrasonic bath.
- Do not use soap solutions containing plasticisers, alcohol-based solvents or abrasive cleaning products to clean the product.

Risk when handling batteries

- Check the batteries regularly. Leaking batteries can cause damage to the product and it could lead to irritation if the leakage makes contact with the skin.
- Remove the batteries from the battery compartment as soon as they are spent or if the product is to be unused for a long period of time.
- Make sure that the batteries are disposed of properly. Do not dispose of batteries in the household waste.

Intended use

The product is intended for magnifying a reading and working area. It is only suitable for indoor use in dry rooms.

Only use the product as described in this manual. Any other use is considered improper and can lead to property damage or personal injury. Do not make any changes to the construction of the product. No liability is assumed for damage caused by improper or incorrect use.

Product description

Illuminated magnifier

A Lens	B [On/Off] button
C Battery compartment	D Stand
E Adapter	

Operation

Inserting / changing the batteries

- Always use 3 new batteries of the same type.
- Note the position of the respective positive terminal.

1. Use the [On/Off] button **B** to switch off the lighting of the illuminated magnifier.
 2. Pull the battery compartment **C** off the illuminated magnifier from above.
 3. Remove the spent batteries from the battery compartment.
 4. Insert the new batteries into the battery compartment.
- ⇒ The negative terminal of the battery is pointed in the direction of the battery compartment's spring.
5. Position the battery compartment on the top edge of the mounting plate and push the battery compartment downwards until it stops.

Switching the illuminated magnifier on and off

1. Press the [On/Off] button **B** to switch on the light.
2. Press the [On/Off] button again to switch off the light.

Note: The light switches off automatically after 30 minutes.

Using the illuminated magnifier

If you normally wear glasses for reading, you do not need to remove them. The illuminated magnifier is designed to also be used together with your reading glasses.

1. Position the illuminated magnifier on the reading material so that the lens is facing you.
2. Switch on the light.
3. Vary the distance between your eyes and the illuminated magnifier until you see the reading material clearly. The correct distance differs for each individual. It depends on the reading glasses used and the adaptability of your eyes. The guide value for the distance is 200 mm.
4. Switch off the light after use.

Adjusting the light intensity

- ✓ The light is switched on.
1. Keep the *[On/Off]* button **B** pressed.
- ⇒ After approx. 3 sec., the light intensity is switched between 100 %, 75 % and 50 %.
2. Let go of the *[On/Off]* button when the required light intensity is reached.

Setting the colour temperature

You can set the following colour temperatures:

- 6000 K: cold white light
 - 4700 K: neutral white light
 - 2800 K: warm white light
- ✓ The light is switched off.
1. Keep the *[On/Off]* button **B** pressed.

⇒ After approx. 3 sec., the colour temperature is switched between 6000 K, 4700 K and 2800 K.

2. Let go of the *[On/Off]* button when the required colour temperature is reached.

Using the adapter

You can use the illuminated magnifier with the adapter supplied for a more comfortable inclined view.

Inserting the adapter

1. Position the illuminated magnifier with the tip of the stand **D** on the adapter.
 2. Carefully press the back of the illuminated magnifier downwards.
- ⇒ The illuminated magnifier latches onto the adapter.

Removing the adapter

1. Grip the adapter with one hand and the stand with your other hand.
2. Pull the back of the adapter downwards and off the stand.

Technical data

Magnification	
Article no. 156514	2.8 x
Light source	
Lamp	LED (risk group 1 to EN 62471)
Power supply	
Battery	3 × Mignon (AA/LR6) / 1.5 V
Ambient conditions	
Operating temperatures	0 °C ... 40 °C

Cleaning and care

1. Make sure that the illuminated magnifier is switched off.
2. Clean the lens with a soft, lint-free cloth, for example, a microfibre cloth.
3. For heavier soiling, e. g. finger-marks, moisten the cloth slightly. Do not use soap solutions containing plasticisers, alcohol-based solvents or abrasive cleaning products.
4. When not in use, keep the illuminated magnifier covered or pull the enclosed microfibre bag over the lens of the magnifier.

Disposal

Disposal of electrical equipment

This product is subject to the currently valid version of the European Directive on Waste Electrical and Electronic Equipment. Do not dispose of the product in the normal household waste bin.

Dispose of the product or parts of it through a certified disposal facility or your municipal disposal facility. Note and comply with the currently applicable regulations. If in doubt, contact your disposal management facility. Dispose if all packaging materials in an environmentally friendly manner.

Disposal of batteries and rechargeable batteries

Do not dispose of batteries and rechargeable batteries in the household waste bin.

To dispose of the device, remove the batteries or rechargeable batteries from the device and hand them in to an official collection point.

Declaration of Conformity

The product meets all requirements of the relevant EU directives and the national regulations. We confirm this with the CE marking. You can retrieve the declaration of conformity for this product on the following website: <http://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>.

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Trade Name: scribolux

Model No.: 156514

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.
22 Shelter Rock Lane
Danbury, CT 06810
United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any in-

interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

Guarantee

Within the parameters of the applicable statutory provisions, Eschenbach Optik GmbH provides a guarantee for the function of the product described in this manual with regard to any faults arising due to manufacturing defects or material defects. In the event of damage due to improper handling, including damage from a fall or impact, no guarantee claims can be made. The guarantee can only be claimed by presenting proof of purchase.

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