Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing

Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

CE
ESCHENBACH

Deutsch.....	3
English.....	6
Français.....	9
Italiano.....	12
Español.....	15
Nederlands.....	18
Dansk.....	21
Svenska.....	24
Norsk.....	27
Suomi.....	30
Polski.....	33
Český.....	36
日本語.....	39

Deutsch

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► Brandgefahr!

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Safety Instructions

▶ **Danger of fire!**

Lenses in optical devices can cause significant damage by the focusing of light to generate heat if handled and stored improperly! Take care that you never leave uncovered optical lenses lying in direct sunlight!

▶ **Danger of blinding or injury!**

Never look directly at the sun or other bright light sources through any optical equipment!

▶ **Protect your monocular against impacts of all kinds, as well as against moisture and excess heat. Never use your monocular in the vicinity of heat sources.**

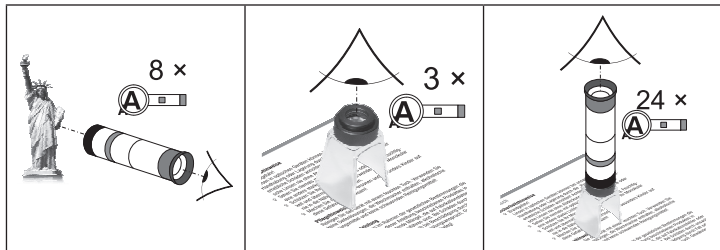
▶ **Ensure that other people, particularly children, are aware of these risks!**

▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Care Instructions

Clean the appliance with a moist cloth. Do not use any soap solutions that contain softeners, alcohol-based solvents, or abrasive cleansers.

Use

The eye cup can be folded over by people wearing glasses.


To focus the image on distant objects, turn the lens-side part of the hand monocular inwards. To focus the image on closer objects, turn the lens-side part of the hand monocular outwards. For use as a standing microscope, turn the front part of the hand monocular inwards and screw the standing magnifier onto the lens side of the hand monocular.

If you would like to hang the hand monocular over your shoulder, attach the supplied cord strap to the lugs on the casing. When the hand monocular is not in use, attach both lens caps and store the monocular in the supplied case.

Warranty

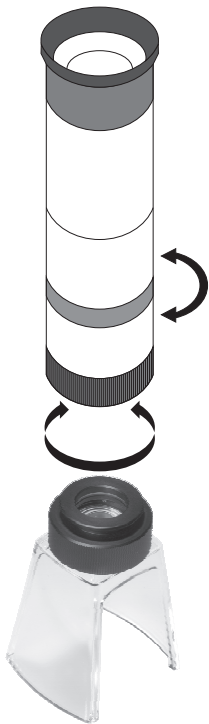Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH