
visomax

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

ESCHENBACH

Deutsch	5
English	6
Français.....	8
Italiano	10
Español	11
Nederlands.....	13
Dansk	15
Svenska	16
Norsk.....	18
Suomi	19
Polski	21
Česky.....	22
日本語.....	24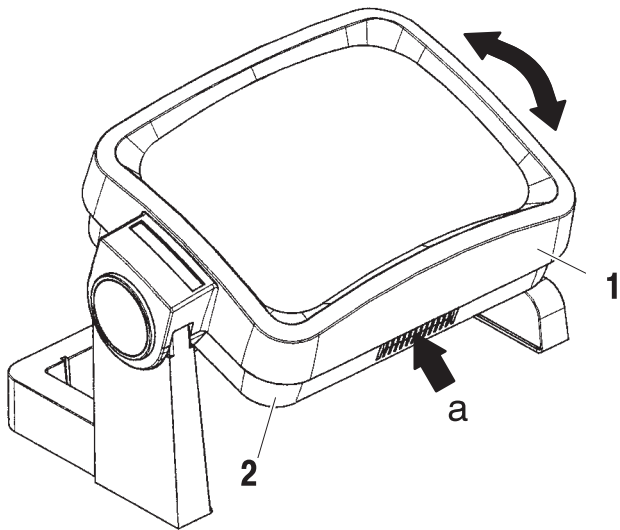

No.: 263611	
	
1 ⇒	7,0 Dpt = 2,8 x
1 + 2 ⇒	11,0 Dpt = 3,8 x

Deutsch

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English

Free-standing reading lenses by Eschenbach. For better quality of life.

You have selected a quality product from the house of Eschenbach which has been produced using the latest manufacturing processes in Germany, a top-class product "Made in Germany". We congratulate you on your choice.

Safety instructions

- **Risk of fire!**

Lenses used in optical equipment may cause considerable damage through their “burning glass effect” if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

- **Danger of blinding or injury!**

Never look at the sun through any optical equipment!

- **Protect your free-standing reading lens against jolts and impact shocks, moisture, and excessive heat. Never place your free-standing reading lens on heaters or in the sunlight!**

- **Ensure that other people, especially children, are aware of these dangers!**

- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

Operating instructions

[1] Main lens

[2] Additional lens (removable),

1. Place the free-standing reading lens on the text you want to read.
2. Tilt the lens to suit your angle of view.

Remove additional lens

1. Place free-standing reading lens on a base.
2. To unlatch the additional lens [2], press in the direction of arrow [a] and remove the additional lens by pulling downward.

○ For reassembly, the additional lens engages under the main lens [1].

Instructions for care

When the free-standing reading lens is not in use, it should be kept covered. Clean the housing with a damp cloth. Clean the lenses using a soft, lint-free cloth (e. g. specta-

cle cloth). If the lens has become very dirty (e.g. fingerprints), clean it carefully with lukewarm water and dry it carefully.

Do not use any soap solutions that contain softeners, alcohol-based solvents, or abrasive cleaning products. They may damage the lenses.

Warranty

We are required by law to guarantee the functionality of the product described in this user manual with regard to defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or bumped, are not covered under the warranty. Warranty benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH