


Firmware Update optaro® XL

Instructions

1. Download the update file from the Eschenbach homepage in the **Materials** section using your iPad:

eschenbach-vision.com/en-GB/products/product/optaror-xl-11-ipad-holder165341

2. Once downloaded, you will find the BRN file in the **Downloads folder** on your iPad.

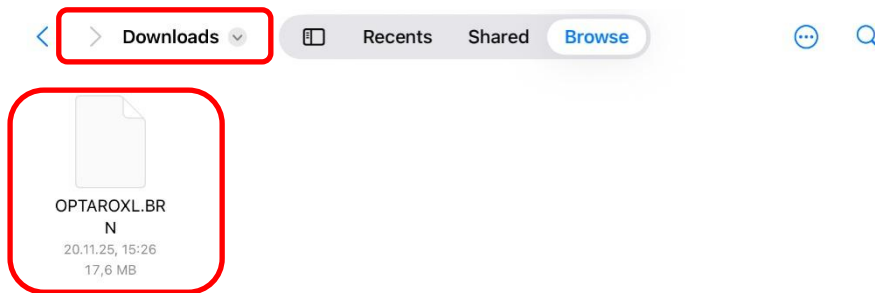


3. Turn on optaro® XL. When the **blue status LED** is lit continuously, press and hold the update button on the back of the device (see red mark) for approx. 5 seconds **until the LED connection indicator** – which is located at the USB-C port and next to the update button – **turns orange**.



As long as the **LED connection indicator is orange** (or flashing), optaro® XL is in update mode and cannot be used as usual. If optaro® XL is switched off during the firmware update or the reset button is pressed, the update will be discontinued and the old firmware will remain in use.


4. Connect optaro® XL and your iPad using the short USB-C cable.



5. Open the **Files** folder on your iPad , then open the **Downloads** folder. The BRN file is stored there.
6. The storage location "**NO NAME**" is displayed on the iPad's sidebar. Move or copy the BRN file into the "**NO NAME**" folder.

There are different ways to do this:

- a. Drag and drop the file from the Downloads folder into the "**NO NAME**" folder.
- b. Tap and hold the file until the context menu appears, then select "**Move**" from the menu at the top and tap the "**NO NAME**" folder. This will move the file to the designated folder.

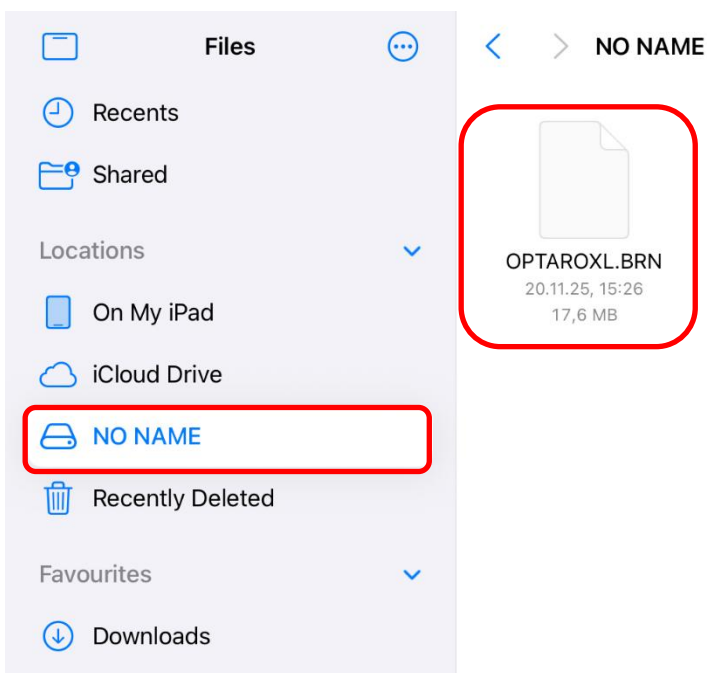

Important:

Only use the update file named **OPTAROXL.BRN**. If the file has been accidentally renamed, duplicated, or downloaded multiple times – resulting in a modified file name – the device will not recognize the update.

- Now unplug the short USB-C cable from the iPad.
The LED will flash orange for a short time – this can take up to 3 minutes.
However, it is also possible that the LED will turn off after a few seconds and then flash again. In this case, wait a little longer to ensure the update completes successfully.

The LED will turn off completely after a while. The new firmware has then been successfully installed on optaro® XL.

- Open the settings of the optaro® XL app to check whether the update has been successfully installed. The **current firmware version** is displayed in the “**About**” section.

